

# New Product Information!



## Solving the puzzle of personal glucose patterns just got easier!

**NEW**

### Introducing the CareLink™ USB upload device!

The new, wireless way to upload information from the MiniMed Paradigm® insulin pump to a personal computer (PC)\*



- Sleek, compact design eliminates external cables
- Fast transmission
- No batteries required
- Windows® compatible

## Medtronic CareLink™ PERSONAL

THERAPY MANAGEMENT SOFTWARE FOR DIABETES

CareLink Personal software is a free web-based program that allows you to discover patterns and problems that meter software and logbooks would not uncover. This one-of-a-kind resource gives you and your healthcare provider clear insight into your personal glucose patterns for improved therapy management.



Compatible insulin pumps:  
• 515/715 • 522/722

For information on CareLink Personal software and the Carelink USB device go to: [www.medtronicdiabetes.com/carelink](http://www.medtronicdiabetes.com/carelink)



CareLink™ USB



**Medtronic**

Alleviating Pain • Restoring Health • Extending Life

# Getting started with CareLink™ USB\* is easy!

1

Visit CareLink Personal software at:  
[www.carelink.medtronicdiabetes.com](http://www.carelink.medtronicdiabetes.com),  
log on and select the "Upload tab".



New users need to enroll before using.

2

Follow the on-screen directions until you reach the "Select the Link Device" screen, select CareLink USB and click "Next".



3

You will be prompted to download the software driver for your new CareLink USB device. Follow the on-screen directions until the driver download is complete.



4

When prompted, plug in your CareLink USB device into an available USB port on your computer and click "Next". Your computer will recognize your CareLink USB device.



**That's it! You're now ready to upload data from your Medtronic device to CareLink Personal software. Simply follow the on-screen directions and CareLink Personal software will walk you through the upload process.**

The CareLink USB upload device replaces the Paradigm Link® Meter and its cable as the new wireless way to upload your MiniMed Paradigm® insulin pump information to CareLink Personal Software.\*\*

To upload data directly from your compatible blood glucose meter to the PC, use the cable provided by that meter company to connect the meter to the PC. Please contact your meter company to learn more.

\* Does not work with ParadigmPAL™ or Solutions® Software

\*\* CareLink Personal software will continue to support the Paradigm Link as an upload device.

CareLink and ParadigmPal are trademarks and Solutions, Paradigm Link, and Paradigm are registered trademarks of Medtronic MiniMed, Inc.  
©2008 Medtronic MiniMed, Inc. All rights reserved. 9402109-011 020108