

I AM WEARING AN INSULIN PUMP. IN AN EMERGENCY, PLEASE CONTACT:

Doctor: _____

Phone number: _____

Family member: _____

Phone number: _____

I have diabetes. If my behavior is peculiar (appear intoxicated) or I am unconscious, it may be a reaction to the insulin.

I am not intoxicated. Call 911 for assistance. If I am able to swallow, give me a source of sugar, (examples: juice, candy, non-diet soft drink).



Medtronic

MINIMED

Medtronic MiniMed

Northridge, CA 91325

USA

800-646-4633 (800-MiniMed)

818-576-5555

www.minimed.com

6024946-0117/03

Healthcare Professionals

I am wearing an external insulin pump.

- It is delivering fast-acting insulin at a constant rate.
- To Suspend insulin delivery, press ACT, use arrow button to select Suspend, press ACT. Follow instructions on the screen.
- DO NOT remove the pump battery.
- DO NOT remove the pump from user without medical consent.
- If the pump is alarming, follow the instructions on the pump screen and/or call the Medtronic MiniMed 24-hour Product Help Line at 800-MiniMed (800-646-4633).

High blood glucose and preventing diabetic ketoacidosis (DKA)

If one blood glucose reading is above 250 mg/dl (13.9 mmol/L):

- ▶ Immediately take a correction bolus
- ▶ Test BG in one hour

If a second blood glucose is above 250 mg/dl (13.9 mmol/L):

- ▶ Take a correction insulin injection by syringe (not through the pump)
- ▶ Check urine for ketones and call healthcare professional if ketones are present
- ▶ Change entire infusion set system. (New reservoir, infusion set, and cannula)
- ▶ Drink liquid with no calories every 30 minutes. (for example: 8 oz. diet ginger ale, broth, water)
- ▶ Test blood glucose every 2 hours and continue to take correction insulin until blood glucose reaches target
- ▶ Call your healthcare professional if your blood glucose and urine ketones remain elevated or you are unable to drink

Low blood glucose (hypoglycemia)

Some symptoms of low BG:

- ▶ Sweating
- ▶ Headache
- ▶ Drowsiness
- ▶ Trembling
- ▶ Blurred vision
- ▶ Hunger
- ▶ Dizziness
- ▶ Confusion
- ▶ Palpitations
- ▶ Speech Impairment

Hypoglycemia protocol: the rule of 15

If blood glucose is 70 mg/dl (3.9 mmol/L) or below:

- ▶ Eat/drink 15 grams of carbohydrate
- ▶ Check BG again in 15 minutes; if not above 70 mg/dl (3.9 mmol/L) repeat treatment
- ▶ Check BG again in 15 minutes; if still not above 70 mg/dl (3.9 mmol/L) repeat treatment and contact your healthcare professional

These items have 15 grams of carbohydrate:

- ▶ Glucose tablets
(READ label: three 5 gram tablets, or four 4 gram tablets)
- ▶ 4 oz. of juice or soda (not diet)
- ▶ 6-7 hard candies such as lifesavers
- ▶ 1 tablespoon of table sugar or honey