CareLink™ PRO

THERAPY MANAGEMENT SOFTWARE FOR DIABETES

Installation Guide / Guide d'installation
Installationsanleitung / Guía de instalación
Installatiehandleiding / Guida all'installazione
Installeringshandbok

Medtronic

© 2007, Medtronic MiniMed. All rights reserved.

Paradigm[®] and Paradigm Link[®] are registered trademarks of Medtronic, Inc.

CareLink[™], Com-Station[™] and ComLink[™] are trademarks of Medtronic, Inc.

BD Logic[™] is a trademark of Becton, Dickinson and Company. Microsoft[®] Windows[®] 2000 and Microsoft[®] Windows[®] XP are registered trademarks of Microsoft Corporation. Adobe[®] and Reader[®] are registered trademarks of Adobe Systems, Incorporated

U.S., international, and foreign patent applications are pending

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CareLink™ Pro is designed to simplify and enhance the management and treatment of diabetic patients. The software transforms insulin pump, glucose meter, and glucose sensor data into actionable information through a series of reports. These reports enable healthcare professionals to make more informed treatment decisions for their diabetic patients.

System requirements

- This application has been validated and approved for use with U.S., UK, German, French, Spanish, Italian, Dutch, and Swedish versions of *Windows XP Professional* (minimum of Service Pack 2).
- Minimum of 512 MB of RAM (1 GB or greater recommended).
- Minimum of 300 MB of free hard disk space for program installation.
- Additional disk space needed for establishing and expanding a patient database. Over time, the database could occupy up to 2GB of space (4GB including backups).
- Minimum screen resolution of 800 x 600 pixels (1024 x 768 recommended).
- Serial communication port available (for connection to Paradigm Link® Monitor*, Com-Station™, ComLink™ and supported meters).
- USB communication port available (for connection to Paradigm Link® Monitor* and CareLink™ USB).
- Keyboard and pointing device (such as a mouse, touch pad or trackball).
- CD-ROM drive for program installation.
- Adobe Reader software (minimum of version 5.0).

* *Not available in all countries.*

Supported devices and hardware accessories

For supported devices and hardware, see the *CareLink® Pro User Guide*. To view the user guide, refer to “Accessing the user guide” on page 6.

CD handling

To protect the non-label side from scratches, fingerprints, dust particles, and smears, never touch the non-label side or place it face down on a hard surface. Avoid exposing the disc to direct sunlight, high temperatures, humidity, dust, or dirt.


Upgrading the software

If you are already using CareLink Pro, you can use this CD to upgrade your application. Follow the steps in this section. If you are installing CareLink Pro for the first time, see “Installing the software.”

NOTE: It is recommended that you back up your database before upgrading. For instructions, see the *CareLink Pro System Administration Guide*, located on the CD-ROM for this software:

`\\program files\\Medtronic\\CareLink\\AppDir\\Documentation\\
SystemAdminGuide7335.pdf`

1. Close all software programs running on your computer.
2. Insert the CareLink Pro installation CD into the CD drive with the label side facing up. The following message is displayed.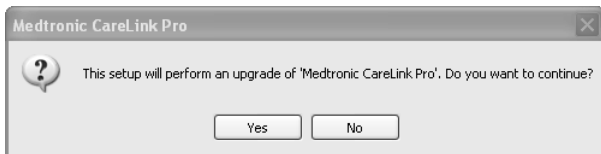

3. Click YES to continue the CareLink Pro upgrade.
4. A message is displayed indicating that the installer will upgrade your database. Click NEXT to continue the upgrade process.
5. You will see the progress of the upgrade. When it is complete, a message is displayed, indicating that CareLink Pro has been successfully installed.
6. Click FINISH to exit the wizard.


Installing the software

These steps are for a typical installation. Always read and follow instructions as they appear on your computer screen.

NOTE: To install this software, you must have administrator privileges for the computer. If you do not, see “Troubleshooting” on page 7.


1. Before proceeding with the installation, close all software programs running on your computer.

2. Insert the CareLink Pro installation CD into the CD drive with the label side facing up.
The Choose Setup Language dialog box is displayed.
3. If the dialog box is displayed, go to step 4. If it is not displayed, follow steps a through c to start the software installation process:
 - a) Click the START button on the Windows desktop and select Run.
The Run dialog box appears.
 - b) Click BROWSE and select your CD drive.
 - c) Highlight the *setup.exe* file, and click OK. The Choose Setup Language dialog box is displayed.
4. Select the desired language and click OK.
If the Welcome screen is displayed, continue to step 5. If not, one or both of the following conditions may apply:
 - a) If components required by CareLink Pro are missing from your system, you will be notified. Click NEXT and follow the screen prompts to allow CareLink Pro to install the missing items now.
 - b) If you are installing CareLink Pro on a system with an operating system other than the validated one, the following message is displayed. Click YES to continue the CareLink Pro installation.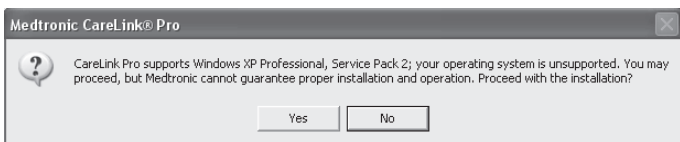

Once the above conditions are met, the Welcome screen will be displayed.

5. Click NEXT.
6. To accept where CareLink Pro files will be installed, click NEXT. To pick a new location, click CHANGE, select a new location, and click NEXT.
The following screen is displayed.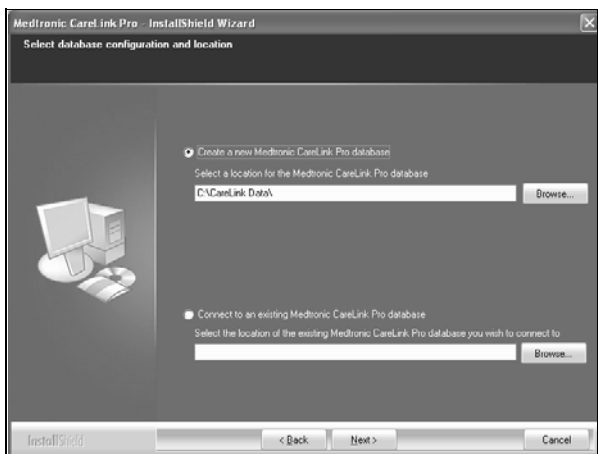

7. Decide whether to establish a new database or connect to an existing one. If you are the first person or only person to install this software, you will likely create a new database.

If you plan to install the software on multiple, networked PCs, you are likely to want all the PCs to connect to a single database. One way to do this, is to create a new database during the first installation of CareLink Pro (in a location that the other PCs can access, such as a network drive). Then, when you install the software on other networked PCs, connect to this existing database during the installation process.


NOTE: To aid in choices about database installment, and other advanced installation decisions, refer to the *CareLink Pro System Administration Guide*, located on the CD-ROM for this software:

`\\program files\\Medtronic\\CareLink\\AppDir\\Documentation\\
SystemAdminGuide7335.pdf`

Follow the steps for the appropriate option.

Option 1: Create a new database

- Click *Create a new Medtronic CareLink Pro database*.
- Click NEXT to accept the default location. Or, click BROWSE to specify the location of the new database (on your PC or on a network drive), and click NEXT. The following screen is displayed:

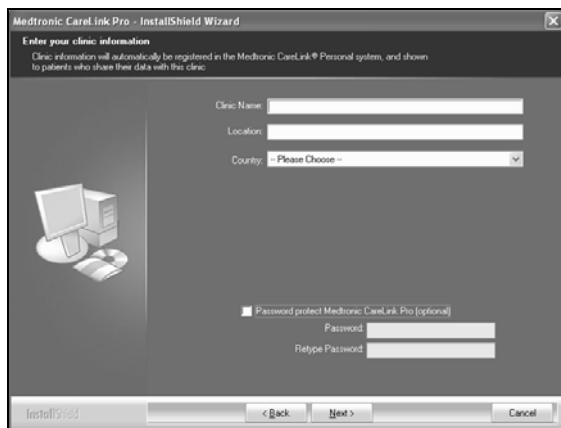- c) Type your clinic's name and location, and click NEXT. This information displays each time you start the software. It is also how patients identify you through their CareLink Personal account.
- d) (Optional) If you want to protect the use of the software with a password, make sure there is a check mark next to *Password Protect Medtronic CareLink Pro*. Then, type the password in both the *Password* and *Retype Password* fields.
If you do not want to require a password, clear the check mark next to *Password Protect Medtronic CareLink Pro*.

Option 2: Connect to an existing database

- a) Click *Connect to an existing Medtronic CareLink Pro database*.
 - b) Click BROWSE and locate the existing database to which you want to connect. The CareLink Pro database file has a filename extension of .cldb. It has a default location, but may be stored in a different directory:
C:\CareLink Data
8. Read the Software License Agreement. Click *I accept the terms of the license agreement*, and then click NEXT to continue.
 9. Click NEXT to continue the installation.
 10. You will see the progress of the installation. When it is complete, the CareLink Pro icon will be displayed on your desktop.
Also, icons are added to the Start menu, including shortcuts to the User Guide and the Report Reference Guide.
 11. Check the box if you want to install Adobe Reader. Click NEXT.

NOTE: If the installation of Adobe Reader is not successful, the CareLink Pro software installation will still complete. See “Installing Adobe Reader from the CD” on page 7.

12. Click **FINISH** to close the installation wizard.
13. If instructed to do so, restart your computer.
14. To run the software, double-click the CareLink Pro icon displayed on your computer desktop, or choose the software from your **Start > Programs** menu.

Accessing the user guide

Once you have installed CareLink Pro, you can view a PDF version of the user guide from the Help menu of the software. Take the following steps:

1. Click the **Help** menu option.
2. Click **View User Guide (PDF)** and select the desired language.

The CareLink Pro user guide opens in the specified language.

Opening the user guide from the CD

To open and view the user guide directly from the CareLink Pro installation CD, take the following steps:

1. Make sure the CareLink Pro installation CD is inserted in your CD drive.
2. Open My Computer from your desktop or the Start menu, and double-click the CD drive icon.
3. Go to the **program files\Medtronic\CareLink\AppDir\Documentation** directory.
4. Double-click the **UserGuide** directory.
5. Double-click the user guide file for the language you prefer.

NOTE: If you cannot access the user guide, you may need to install Adobe Reader. See the next section.

Installing Adobe Reader from the CD

To install Adobe Reader directly from the CareLink Pro CD, take the following steps:

1. Close the CareLink Pro software.
2. Insert the CareLink Pro installation CD into the CD drive.
3. Open My Computer from your desktop or the Start menu, and double-click the CD drive icon.
4. Double-click one of the following Adobe Reader files:
 - **AdbeRdr80_en_US.exe** (English)
 - **AdbeRdr80_de_DE.exe** (German)
 - **AdbeRdr80_fr_FR.exe** (French)
5. Follow the on-screen instructions for installation.
6. If prompted, restart the computer.

NOTE: If you encounter problems installing Adobe Reader, refer to the Adobe Web site at www.adobe.com.

Troubleshooting

Below are some problems you may encounter when installing this software, and how to handle them.

No Administrator rights on this computer

If you receive an error message saying that you do not have administrator privileges for this computer, you can do one of the following:

- Install the software using a different user account that has administrator privileges for this computer.
- Contact your computer support department, and have them either change your privileges or install the software for you.

Cannot create a new database

You will not be allowed to create a new database in a directory that already contains a CareLink Pro database. You must do one of the following:

- Create a new database in a different directory from the one containing the existing database.
- Move the existing database to a different location on the computer, and then install the software.

Assistance

For guidance with installation and software problems, please contact one of the following, as appropriate for your region:

U.S. and Canada

Medtronic MiniMed provides a number to call for guidance with installation and software problems. The hours to call are Monday through Friday from 5 a.m. to 6 p.m. and Saturday from 8 a.m. to 5 p.m. Pacific Standard Time.

Department	Telephone number
24 Hour HelpLine within the United States and Canada	(800) 646-4633 800-MiniMed
24 Hour HelpLine outside the United States and Canada	818-576-5555
MiniMed Web site	www.minimed.com

Outside the U.S. and Canada

For software or installation assistance, please contact your local country representative.

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Medtronic B.V.