
MiniLink™ REAL-Time

TRANSMITTER

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
The Medtronic MiniLink[®] transmitter (MiniLink) is a component of select continuous glucose sensing systems and sensor-enabled insulin pump systems. The MiniLink powers the glucose sensor, collects glucose data, and wirelessly sends the data to a monitor. The monitor is a Medtronic continuous glucose sensing monitor or Medtronic MiniMed[®] sensor-enabled insulin pump.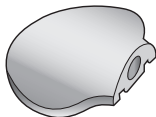

MiniLink (transmitter)

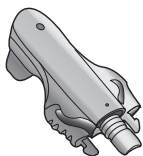MiniLink tester

MiniLink charger

A complete MiniLink kit includes:

- MiniLink (MMT-7703)
- MiniLink tester (MMT-7706)
- Sen-serter[®]
- MiniLink charger (MMT-7705 OR MMT-7715)
- AAA or LR-03 alkaline battery(ies)
- Occlusive dressing

Indications for use

The MiniLink is indicated for use as a component of select Medtronic continuous glucose sensing systems and Medtronic MiniMed sensor-enabled pump systems.

Contraindications

Do not expose your MiniLink to MRI equipment or other devices that generate strong magnetic fields. If your MiniLink is inadvertently exposed to a strong magnetic field, discontinue use and contact your physician.

Warnings

Product contains small parts and may pose a choking hazard for young children.

Do not use the MiniLink to send glucose readings to the monitor while onboard an aircraft. Disconnect the MiniLink from the sensor while travelling on an aircraft or if interference with other transmitting devices occurs.

The sensor should be removed if redness, bleeding, pain, tenderness, irritation, or inflammation develops at insertion site, or if you experience unexplained fever.

An optional occlusive dressing should be removed if irritation or reaction to the tape develops.

The sensor may create special needs regarding your medical conditions or medications. Discuss with your doctor before using the sensor.

Wait 5 minutes after sensor insertion before connecting the transmitter to the sensor.

- Make sure that the site is not bleeding before connection.
- If bleeding occurs, apply steady pressure with a sterile gauze or clean cloth at the insertion site until bleeding stops. After bleeding stops, attach the MiniLink to the sensor.
- If bleeding persists after 3 minutes, remove the sensor and discard. Insert a new sensor in a different location.

Contact the HelpLine or your local representative listed on the enclosed International Contact card if you experience any adverse reactions associated with the MiniLink or sensor.

Note: *The MiniLink is designed to withstand common electromagnetic interference, including airport security systems.*

Precautions

Establish a rotation schedule for choosing new sensor sites. Avoid sites that are constrained by clothing, have scar tissue, or are subject to rigorous movement during exercise.

FCC notice

This device complies with the United States Federal Communications Commission (FCC) and international standards for electromagnetic compatibility.

This device complies with Part 15 Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesirable operation.

The MiniLink does not interfere with any radio frequency signals transmitted from outside sources. These FCC standards are designed to provide reasonable protection against excessive radio frequency interference and prevent undesirable operation of the device from unwanted electromagnetic interference.

Important: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

RF Interference from other devices

Common consumer electronic devices that transmit in the same frequency band used by the MMT-7703 MiniLink transmitter may prevent the receiving device (Guardian[®] REAL-Time monitor or MiniMed Paradigm[®] REAL-Time insulin pump) from receiving the glucose information sent by the transmitter. Most cellular (mobile) phones and 900 MHz cordless phones, when transmitting or receiving, may cause significant interruption of transmitter-receiver communication. It is likely that other devices operating in similar frequency ranges will have a similar effect. This interference, however, will not cause any incorrect data to be sent and will not cause any harm to your MiniLink.

Guardian REAL-Time monitors and MiniMed Paradigm REAL-Time insulin pumps include a programmable "Weak Sensor" alert that notifies you when one or more expected MiniLink transmissions were not received as expected by the receiving device. (The receiving device will also issue a "Lost Sensor" alert if communication is interrupted for approximately 40 minutes.)


Communication problems can typically be resolved by ensuring that the distance between transmitter and receiving device is less than 6 feet (1.8 meters), and by turning off or moving away from other RF transmitting devices. You can also reorient or relocate the MiniLink and/or the receiving device to try to correct the interference. Testing conducted with several different cellular phones suggests that interference will not be a problem if the phone is at least 12 inches (31 cm) from the transmitter or receiving device while it is being used (greater separation distance may be required for certain devices).

MiniLink charger

The MiniLink contains a non-replaceable, rechargeable battery that you can recharge as needed with the charger. The charger has a green light that shows the charging status and a red light that communicates any problems during charging. If you see a red light, see the Troubleshooting section. The charger needs a AAA or LR-03 battery to operate.



Installing a new charger battery (Blue Charger MMT-7705)

1. Remove the charger battery cap by turning it counter-clockwise 1/4 turn using a coin in the groove of the cap.
2. Insert a new AAA or LR-03 battery with the flat (-) end first. Make sure that you align the small bumps on the battery cap with the small notches in the charger's battery opening. Push in the cap all the way using a coin. Turn the cap clockwise 1/4 turn to close.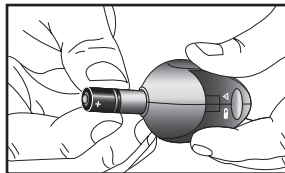

Installing a new charger battery (Gray Charger MMT-7715)

1. Push the battery cover in and slide it off (as shown in the illustration).
2. Insert a new AAA or LR-03 battery. Make sure the (+/-) symbols on the battery align with the symbols shown on the charger.
3. Slide the battery cover back on the charger.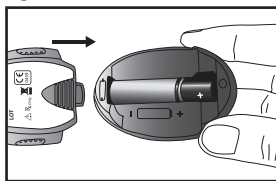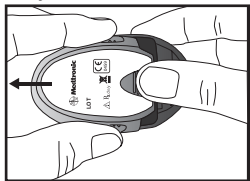


Note: A new AAA or LR-03 battery contains enough power to recharge the MiniLink more than 40 times. If the battery is installed incorrectly or is low, the charger will not work. Repeat the battery installation steps using a new battery.


Charging the MiniLink

Before using the MiniLink for the first time, you must fully charge the MiniLink battery which may take up to 8 hours. It is recommended to recharge the MiniLink after each sensor use. If you choose to recharge the MiniLink after a 3-day sensor use, the charging time will be less than 20 minutes. A fully charged MiniLink battery will work more than 14 days without recharging. After 14 days of use, the MiniLink will fully recharge in less than 2 hours.


1. If a green light on the MiniLink is lit or flashing, do not connect it to the charger. The MiniLink will not charge with its green light on. Wait for the green light to turn off (approximately 30 seconds), then connect the MiniLink to the charger.
2. Connect the MiniLink to the charger by lining it up, flat side down, with the charger. Push the two components together fully.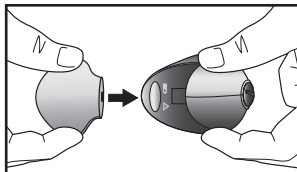

3. Within 10 seconds after the MiniLink is connected, a green light on the charger will flash for 1-2 seconds as the charger powers on. For the rest of the charging time, the charger's green light will continue to flash in a pattern of four flashes with a pause between the four flashes.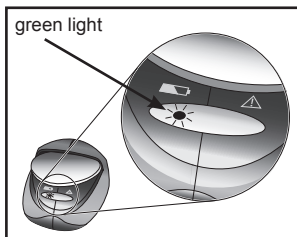


4. When charging is complete, the green light on the charger will stay on, without flashing, for 15-20 seconds and then turn off.
5. After the green charger light turns off, disconnect the MiniLink from the charger. The green light on the MiniLink will flash for about 5 seconds and then turn off.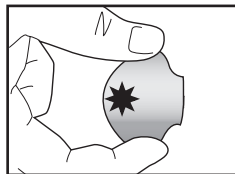

Setting up the MiniLink

The serial number is on the flat side of the transmitter, as shown on the right. See your monitor user guide for steps on entering this number.


SN GT2XXXXXXM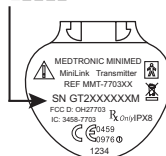

Connecting to the sensor

The MiniLink must be fully charged before connecting it to the sensor.

1. After the sensor is inserted (follow the instructions provided with your sensor), wait 5 minutes before connecting the transmitter. Check for bleeding. Make sure that bleeding has stopped (see Warnings page for details), then connect the MiniLink to the sensor.
2. Touch the inserted sensor at back of assembly to prevent movement.
3. Hold the MiniLink as shown to line up the two notches on both sides with the flexible side arms of the sensor. The MiniLink flat side with the label should face the skin.
4. Slide the MiniLink onto the sensor and push it in firmly until the flexible side arms of the sensor “click” into the notches on both sides of the MiniLink. In the next 20 seconds the MiniLink will flash green for about 10 seconds with a proper connection.
5. If the MiniLink light does not flash, disconnect it from the sensor, wait for several seconds and then reconnect. If the MiniLink light still does not flash, charge the MiniLink.
6. After the MiniLink light flashes green, use your monitor to start communicating with the sensor and to start the sensor initialization (see your monitor user guide for instructions).
7. After the MiniLink successfully sends signals to the monitor, you may choose to put an occlusive dressing over the MiniLink and the sensor.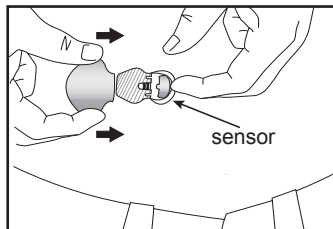


Bathing and swimming

After the MiniLink and sensor are connected, they form a water-tight seal to a depth of 8 feet (2.4 meters) for up to 30 minutes. You can shower and swim without removing them. No occlusive dressing is needed.

Disconnecting from sensor

1. Hold the MiniLink as shown, and pinch the flexible side arms of the sensor between your thumb and forefinger.
2. Gently pull the MiniLink away from the sensor assembly.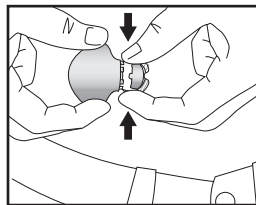


MiniLink tester

The blue tester is used to test the MiniLink and to make sure it is working.

Connecting the tester


1. Hold the MiniLink and the tester as shown. Line up the flat side of the tester with the flat side of the MiniLink.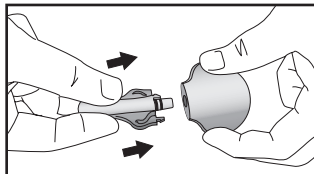

2. Push the tester into the MiniLink until the flexible side arms of the tester click into the notches on both sides of the MiniLink.
3. Within 20 seconds the green light on the MiniLink will flash for about 10 seconds when properly connected.
4. Check the sensor icon on the monitor to ensure that the MiniLink is sending a signal (see the monitor user guide).

Disconnecting the tester


1. Hold the MiniLink body as shown and pinch the side arms of the tester.
2. With the tester arms pinched, gently pull the MiniLink away from the tester.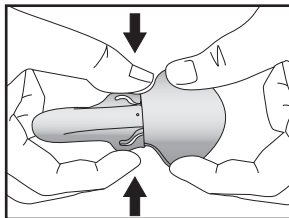

Note: To save MiniLink battery life, do NOT leave the tester connected after testing.


Troubleshooting

Question: Why did the flashing green charger light turn off and a longer flashing red charger light turn on during charging?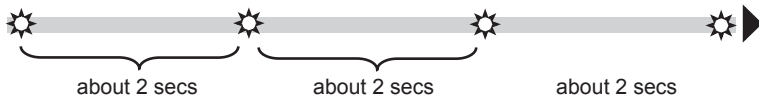


Answer: The MiniLink battery is very low. Leave the MiniLink on the charger for 8 hours to completely recharge. If the red light is still flashing after 8 hours, call the HelpLine or your local representative. It may be time to replace your MiniLink.

Question: Why do I see quick flashing red lights on the charger?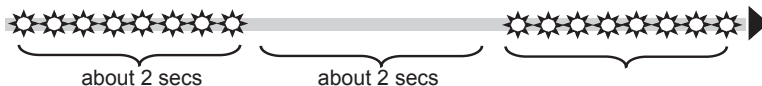

Answer: Your charger battery is low. Make sure the MiniLink is not connected to the charger and replace the charger battery with a new AAA or LR-03 battery.

Question: Why do I see a mix of quick and long flashing red lights on the charger?

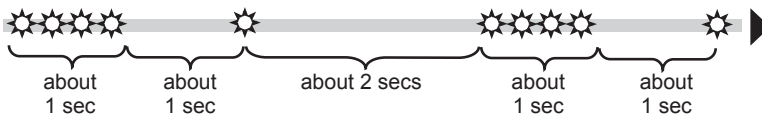Answer: Your charger AND MiniLink batteries are very low. Replace the charger's AAA or LR-03 battery. If you now get the pattern for very low MiniLink battery, leave the MiniLink on the charger for 8 hours to recharge. If the red light is still flashing after 8 hours, call the HelpLine or your local representative. It may be time to replace your MiniLink.

Question: I had my MiniLink on the charger for a day. Will this damage my MiniLink?

Answer: It will not damage the MiniLink. You cannot overcharge it.

Question: What should I do if the MiniLink green light does not flash when connected to the sensor?

Answer: Is the sensor inserted in the body? If **it is not inserted**, the MiniLink will not flash green or send signals to a monitoring device.

If the **sensor is inserted** in the body, you need to disconnect the MiniLink from the sensor, wait for several seconds and then reconnect. If the green light still does not flash, charge the MiniLink.

Question: Why didn't I see the MiniLink green light flash after connecting it to the tester?

Answer: Check the connection. If you still do not see a green light flash, fully recharge the MiniLink battery. Test the MiniLink with the tester. If you still do not see a green light flash, call the HelpLine or your local representative. It may be time to replace your MiniLink.

Cleaning the MiniLink

Caution: Do NOT discard the MiniLink in a medical waste container or otherwise subject it to incineration. MiniLink contains a battery which may explode upon incineration.

For single-patient use

1. Wash your hands thoroughly.
2. Attach the tester to the MiniLink to make sure that NEITHER water NOR disinfectant NOR alcohol gets inside the connector.
3. Dampen a clean cloth with mild liquid soap and warm water. Wipe the outside of the MiniLink.
4. Rinse the MiniLink under warm tap water but do NOT get water inside the connector. If you get water inside the connector, shake the water out and allow it to air dry.
5. Using an antibacterial hand-sanitizer (available at a local drugstore) on a clean, dry cloth, wipe the MiniLink's surface. Do NOT get any hand-sanitizer inside the connector. Repeated exposure to the hand-sanitizer could damage the connectors and affect the MiniLink's performance as a result. If you get hand-sanitizer inside the connector, allow it to air dry.
6. Disconnect the tester from the MiniLink.

7. Place the MiniLink on a clean, dry cloth and air dry for 2-3 minutes.

For multiple-patient use

Warning: Always clean the MiniLink after removing it from the patient and before attaching it to the charger. If there is any blood inside the connector, the MiniLink must be discarded.

Caution: Dispose the MiniLink according to the local regulations for battery disposal (non-incineration).

Follow this procedure to clean the MiniLink after each patient use.

1. Wash your hands thoroughly.
2. Attach the tester to the MiniLink to make sure that NEITHER water NOR disinfectant NOR alcohol gets inside the connector.
3. Dampen a clean cloth with a mild liquid soap solution. Wipe the outside of the MiniLink.
4. Rinse the MiniLink under warm tap water but do NOT get water inside the connector. If you get water inside the connector, shake the water out and allow it to air dry.
5. Apply 3-4 drops of a quaternary ammonium compound disinfectant (for example, Cavicide®) on a clean, dry cloth and wipe the MiniLink.
6. Hold the MiniLink and rinse with 70% Isopropyl alcohol.
7. Disconnect the tester from the MiniLink.
8. Place the MiniLink on a clean, dry, non-shedding cloth and air dry.
9. Once it is completely dry, place it in a sealed bag labeled with the cleaning date (see label example below).

Example: Multiple-use cleaning label

Device:
Date:
Method of Decontamination:
Wash:
Disinfectant:

Cleaning the MiniLink charger or tester

Caution: The charger and tester are NOT watertight. Do NOT immerse in water or any other cleaning agent. If liquid gets inside the black connector of the tester, shake the liquid out and allow to air dry.

For single-patient use

1. Use a damp cloth with mild cleaning solution, such as Dial[®] Dishwashing detergent, to clean any dirt or foreign material from the outside of the charger or tester. Never use organic solvents such as paint thinner or acetone to clean the charger or tester.
2. Apply three to four drops of a quaternary ammonium compound disinfectant (for example, Cavicide[®]) on a clean, dry cloth and wipe all the surface areas of the charger or tester.
3. Slightly dampen a clean cloth with isopropyl alcohol and wipe all the surface areas of the charger or tester.
4. Place the charger or tester on a clean, dry, non-shedding cloth and allow to air dry.

For multiple-patient use

Warning: Always clean the MiniLink after removing it from the patient and before attaching it to the charger. If blood comes in contact with any surface areas of the charger or tester, the contaminated device must be discarded. The charger contains a battery which may explode upon incineration.

Caution: Dispose the charger according to the local regulations for battery disposal (non-incineration).

Follow this procedure to clean the charger or tester after each patient use.

1. Use a damp cloth with mild cleaning solution, such as Dial[®] Dishwashing detergent, to clean any dirt or foreign material from the outside of the charger or tester. Never use organic solvents such as paint thinner or acetone to clean the charger or tester.
2. Apply three to four drops of a quaternary ammonium compound disinfectant (for example, Cavicide[®]) on a clean, dry cloth and wipe all the surface areas of the charger or tester.
3. Slightly dampen a clean cloth with isopropyl alcohol and wipe all the surface areas of the charger or tester.
4. Place the charger or tester on a clean dry, non-shedding cloth and allow to air dry.
5. Once it is completely dry, place it in a sealed bag labeled with the cleaning date (see label example below).

Example: Multiple-use cleaning label

Device:
Date:
Method of Decontamination:
Wash:
Disinfectant:

Storage

Store the MiniLink, charger and tester in a clean, dry location at room temperature. Although not required, you may store the MiniLink on the charger. If the MiniLink is not in use, you must charge the MiniLink at least once every 60 days.

Specifications

Biocompatibility	MiniLink: Complies with ISO 10993-1 for body contact
Applied parts	Transmitter Sensor
Operating conditions	MiniLink Temperature: +32° to +122°F (0° to 50°C) Caution: When operating the transmitter on a tester in air temperatures greater than 41°C (106°F), the temperature of the transmitter may exceed 43°C (109°F). MiniLink Relative Humidity: 10% to 95% with no condensation Charger Temperature: +50° to +104°F (10° to 40°C) Charger Relative Humidity: 30% to 75% with no condensation

Storage conditions	MiniLink Temperature: -4° to +131°F (-20° to +55°C) MiniLink Relative Humidity: 10% to 100% with condensation Charger Temperature: +14° to +122°F (-10° to +50°C) Charger Relative Humidity: 10% to 95% with no condensation
Battery life	MiniLink: 14 days of continuous glucose monitoring immediately following a full charge Charger: Completes 40 typical charging operations using a new AAA or LR-03 battery
Transmitter frequency	MMT-7703NA 916.5 megahertz MMT-7703WW 868.35 megahertz

Warranty

Medtronic MiniMed warrants the Medtronic MiniLink to the purchaser of the product against defects in material and workmanship for a period of six (6) months and the MiniLink charger for up to six (6) months from the date of purchase.

During the warranty period, Medtronic MiniMed will repair or replace, at its discretion, any defective MiniLink or MiniLink charger, subject to the conditions and exclusions stated herein. This warranty applies only to new devices. In the event a MiniLink or charger is repaired or replaced, the warranty period will not be extended past its original expiration date.

This warranty is valid only if the Medtronic MiniLink or charger is used in accordance with the manufacturer's instructions. Without limitation, this warranty will not apply:

- If damage results from changes or modifications made to the MiniLink or charger by the user, or third parties, after the date of sale;
- If service or repairs are performed by any person or entity other than the manufacturer;
- If damage results from a Force Majeur or other event beyond the control of the manufacturer;
- If damage results from negligence or improper use, including but not

limited to: improper storage, submersion in fluid, physical abuse (such as dropping); or

- If blood or water has entered the inside of the MiniLink connector.

This warranty shall be personal to the original user. Any sale, rental or other transfer or use of the product covered by this warranty to or by a user other than the original user shall cause this warranty to immediately terminate. This warranty does not apply to Glucose Sensors and other accessories.

The remedies provided for in this warranty are the exclusive remedies available for any defects in material or workmanship in the product. Any statutory rights granted to consumers under any applicable legislation are reserved. Neither Medtronic MiniMed nor its suppliers or distributors shall be liable for any incidental, consequential, punitive or special damages of any nature or kind caused by or arising out of a defect in the product.

All other warranties, except any applicable mandatory statutory warranties, expressed or implied, are excluded and specifically disclaimed, including, but not limited to, any warranty of merchantability or fitness for a particular purpose.

Icon table

	Serial number
REF	Catalogue number
(1X)	One per container/package
	Date of manufacture
	Manufacturer
	Follow instructions for use
	Fragile product
	Storage temperature range
	European conformity. This symbol means the device fully complies with MDD93/42/EEC (NB 0459).

	European conformity. This symbol means the device complies with R&TTE1999/5/EC directives (NB 0976). Applies to 868.35 MHz device (WW configuration).
	Signifies compliance to Australian EMC and Radio communications requirements
	Radio communication device
CONF	Configuration
	Type BF Device (protection from electrical shock)
IC	Signifies compliance with Industry Canada EMC and radio communications requirements.
IPX8	MiniLink: Protected against the effects of continuous immersion in water (8 feet or 2.4 meters immersion for 30 minutes)
	Attention: Read all warnings and precautions in instructions for use.
	Storage humidity range

© 2008, Medtronic MiniMed, Inc. All rights reserved.

Guardian®, MiniLink®, MiniMed®, Paradigm®, and Sen-serter® are registered trademarks of Medtronic MiniMed. Cavicide® is a registered trademark of Metrex.

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United Kingdom: Medtronic Ltd.
Medtronic Österreich GmbH
Medtronic B.V.