



Medtronic
mio®

INFUSION SET

Infusion set • Cathéter • Infusionsset • Equipo de infusión • Infusieset • Set di infusione •
Slangesett • Infusiosetti • Infusionsset • Infusionssæt • Conjunto de infusão •
Infuzijski set • Infuzni set • Infúziós szerelék • Zestaw infuzyjny • Infúzyon
seti • Infúzna sada • Σετ έγχυσης • Инфузионный набор • 输液器 •
أداة التمشيب • סט עירוי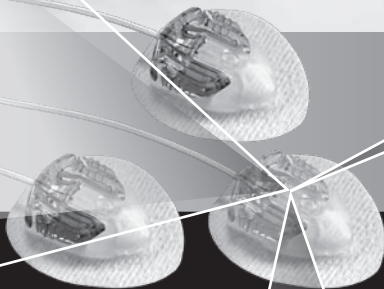





Medtronic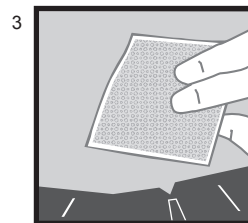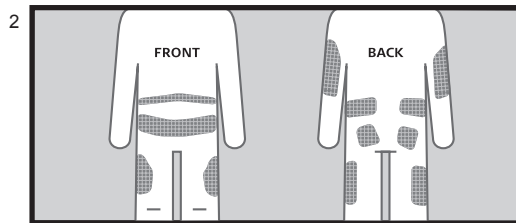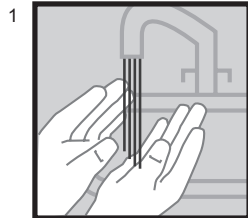


Front • Avant • Körpervorderseite • Parte delantera • Voorkant • Davanti • Foran • Edessä • Fram • Forside • Frente • Spredaj • Čelní strana • elölnézet • Prząd • Ön • Vpredu • Εμπρός • Передняя сторона • 前側 • الخلف • גב

Back • Arrière • Körperrückseite • Parte trasera • Achterkant • Dietro • Bak • Takana • Bak • Bagside • Costas • Zadaj • Zadní strana • hátulnézet • Тыл • Arka • Vzadu • Πίσω • Задняя сторона • 后側 • الامام • חיזית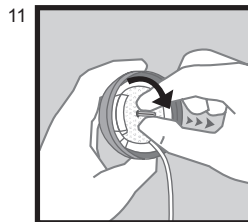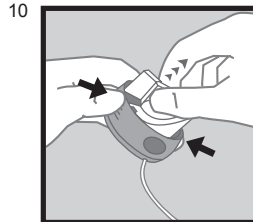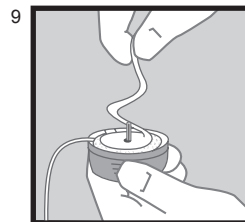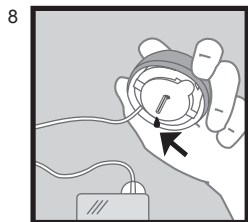

Click • Clic • Klicken • Clic • Vastklikken • Clic • Klikk • Napsahdus • Klick • Klik • Clique • Klik • Kliiknuti • kattanáš • Klik • Tik • Kliiknutie • Κάντε κλικ • Щелчок • 喀哒 • קליק • طغفنة


4

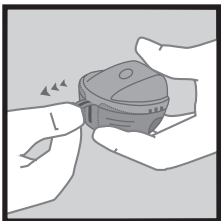
5


6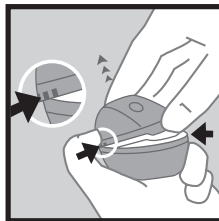


7

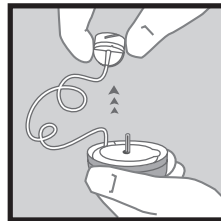
12

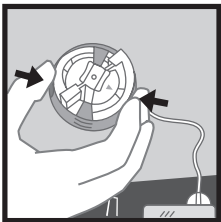
13

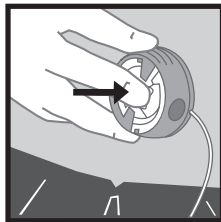14


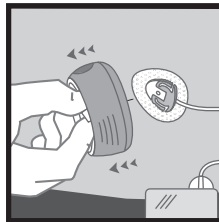
15

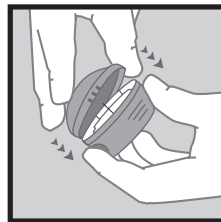Click • Clic • Klicken • Clic • Vastklikken • Clic • Klikk •
 Napsahdus • Klick • Klik • Clique • Klik • Kliknutí •
 kattanáš • Klik • Tik • Kliknutie • Κάντε κλικ • Щелчок •
 喀哒 • קליק • نطقة




DISCONNECTING • DÉCONNEXION • A
 TRENNEN • DESCONEXIÓN •
 LOSKOPPELEN • SCOLLEGAMENTO •
 KOBLE FRA • IRROTTAMINEN •
 BORTKOPPLING • FRAKOBLING •
 DESCONNECTAR • ODKLÓP •
 ODPOJENÍ • LEVÁLASZTÁS •
 ODŁĄCZANIE • ΒΑΓΛΑΝΤΙΥΙ ΚΕΣΜΕ •
 ODPOJENIE • ΑΠΟΣΥΝΔΕΣΗ •
 ОТСОЕДИНЕНИЕ • 断开连接 •
 ניתוק • الفصل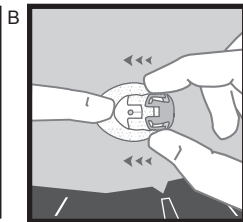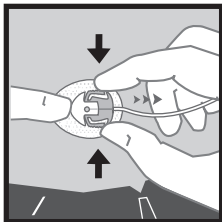


Click • Clic • Klicken • Clic • Vastklikken • Clic • Klick •
 Napsahdus • Klick • Klik • Clique • Cliq • Klik • Kliknutí • kattanás •
 Klik • Tik • Kliknutie • Κάντε κλικ • Щелчок • 喀哒 • קליק • طعنة

RECONNECTING • RECONNEXION • C
 WIEDERANSCHLIEßEN •
 RECONEXIÓN • AANSLUITEN •
 RICOLLEGAMENTO • KOBLE TIL
 IGJEN • KYTKEMINEN TAKAISIN •
 ÅTERANSLUTNING • GENTILSLUTNING
 • VOLTAR A CONECTAR • PONOVNI
 PRIKLOP • OPÉTOVNÉ PŘIPOJENÍ •
 ÚJRACSATLAKOZTATÁS •
 PONOWNE PODŁĄCZANIE •
 YENİDEN BAĞLAMA • OPÁTOVNÉ
 PŘIPOJENIE • ΕΠΑΝΑΣΥΝΔΕΣΗ •
 ВОССТАНОВЛЕНИЕ СОЕДИНЕ-
 НИЯ • 重新连接 • חיבור מחדש •
 إعادة التوصيل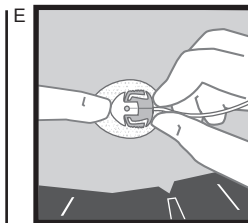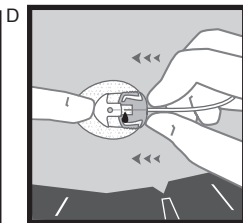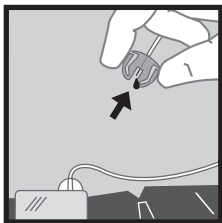

Click • Clic • Klicken • Clic • Vastklikken •
 Clic • Klick • Napsahdus • Klick • Klik •
 Clique • Klik • Kliknutí • kattanás • Klik •
 Tik • Kliknutie • Κάντε κλικ • Щелчок •
 喀哒 • קליק • طعنة

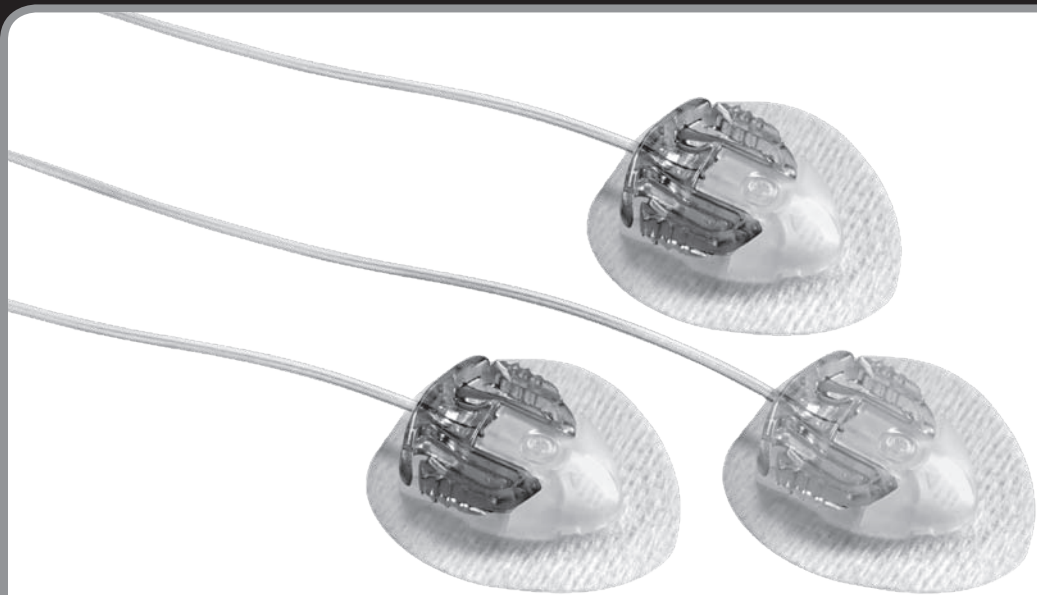Medtronic mio[®]

INFUSION SET

EN See page 8

FR Voir page 12

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ES Consultar la página 20

NL Zie blz. 24

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NO Se side 32

FI Katso sivu 36

SV Se sidan 40

DA Se side 44

PT Consulte a página 48

SL Glejte stran 52

CS Viz strana 56

HU Lásd: 60. oldal

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TR Bkz. sayfa 68

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Indications for use

This product is indicated for the subcutaneous infusion of insulin from an infusion pump.

CONTRAINDICATIONS

This product is neither intended nor indicated for intravenous (IV) infusion or the infusion of blood or blood products.

WARNINGS

- Inaccurate medication delivery, infection and/or site irritation may result from improper insertion and maintenance of the infusion site.
- Ensure sterility by checking that the sterile paper and tamper-proof seal are not damaged.
- Before insertion, clean the insertion site with isopropyl alcohol.
- Remove the needle guard before inserting the infusion set.
- If using this infusion set for the first time, do the first set-up in the presence of your healthcare professional.
- Do not leave air in the infusion set. Prime completely.
- Check frequently to make sure the soft cannula remains firmly in place as you may not feel pain if it pulls out. The soft cannula must always be completely inserted to receive the full amount of medication.
- Release the tubing with caution as a hard pull of the tubing can result in damage to the infusion set/introducer needle. Ensure that the infusion set is properly in place when the tubing is fully released.
- If the infusion site becomes inflamed, replace the set, and use a new site until the first site has healed.
- Replace the infusion set if the tape becomes loose, or if the soft cannula becomes fully or partially dislodged from the skin.
- Replace the infusion set every 48-72 hours, or per your healthcare professional's instructions.
- Do not reinsert the introducer needle into the infusion set. Re-insertion may cause tearing of the soft cannula, which may result in unpredictable medication flow.
- Never prime the set or attempt to free a clogged line while the set is inserted. You may accidentally inject too much medication.
- Do not put disinfectants, perfumes, or deodorants on the infusion set as these may affect the integrity of the set.
- Dispose of the infusion set and introducer needle safely, in a sharps container, after a single use. Do not clean or re-sterilize.
- This device is sterile and non-pyrogenic unless the package has been opened or damaged. Do not use if the package has been opened or damaged.
- For Paradigm users, do not use the infusion set if the tubing connector needle has been damaged.
- Store infusion sets in a cool, dry place. Do not leave infusion sets in direct sunlight or inside a vehicle.
- Use aseptic techniques when temporarily disconnecting the set and consult your healthcare professional on how to compensate for missed medication when disconnected.
- If infusing insulin, carefully monitor your blood glucose levels when disconnected and after reconnecting.
- If infusing insulin, and your blood glucose level becomes unexplainably high, or an occlusion alarm occurs, check for clogs and/or leaks.
- If in doubt, change the infusion set because the soft cannula may be dislodged, crimped and/or partially clogged. Should any of these problems arise, make a plan with your healthcare professional for rapidly replacing insulin. Test your blood glucose level to


- make sure the problem is corrected.
- Never point a loaded insertion device towards any body part, where insertion is not desired.
- If infusing insulin, do not change the infusion set just before bedtime unless you can check your blood glucose 1-3 hours after insertion.
- Reuse of the infusion set may cause damage to the cannula/needle and lead to infection, site irritation, and/or inaccurate medication delivery.

Warranty

- For product warranty information, please contact your local help line, representative, or visit www.medtronicdiabetes.com/warranty.

Preparation and insertion

This product is an infusion device which combines an infusion set (catheter) with an aid for insertion (insertor). This single-use, sterile product is delivered assembled and ready for use. Ensure sterility by checking that the sterile paper and tamper-proof seal are not damaged. Carefully examine the instructions for use before inserting this product. Refer to pump and reservoir user guides for installation and priming. Please see the corresponding illustrations on pages 2 and 3.

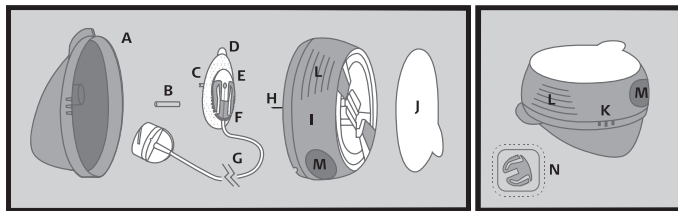Contents

- A. Lid
- B. Needle guard
- C. Soft cannula
- D. Adhesive
- E. Cannula housing
- F. Connector needle
- G. Tubing

- H. Introducer needle
- I. Insertion device
- J. Sterile paper

Important note!

- K. Three raised dots – press here when lifting the lid off (see Figure 6)
- L. Lined indentations – press here when loading (see Figure 10)
- M. Round indentations – press here when releasing (see Figure 12)
- N. Disconnect cover (see Figure B)

Instructions

- 1 Wash your hands.
- 2 Recommended injection areas. Do not use an area right next to your previous insertion site.
- 3 Clean the insertion area with a disinfectant as directed by your healthcare professional. Be sure the area is dry before inserting the infusion set.
- 4 Pull the red tape to remove the protective seal.
- 5 Remove the sterile paper.
- 6 With one hand hold the bottom of the insertion device while pressing the three raised dots on each side of the lid firmly with the other hand. Lift up the lid.
CAUTION: Be careful not to bend or touch the introducer needle when preparing the inserter and infusion set.
- 7 **Unwinding the tubing:** Carefully twist the beginning of the tubing out of the slot. Pull up gently. Now unwind the tubing from the insertion device by pulling it carefully upwards.
Caution: Do not touch the introducer needle when unwinding the tubing. Be careful not to pull too hard when the tubing is fully released. You could accidentally remove the infusion set from the introducer needle.
IMPORTANT NOTE! If the infusion set is not

securely placed in the insertion device with the needle pointing straight ahead prior to insertion, pain or minor injury may occur. To avoid this, please ensure that the following steps are taken gently so the infusion set is not loosened / removed by accident.

- 8** Connect tubing to a filled reservoir. Prime infusion set. Ensure there are no air bubbles in the reservoir or tubing before loading into the pump.

Caution: When priming the infusion set, hold the set with the needle pointing down to ensure that medication does not come in contact with the adhesive backing paper.

- 9** Pull up gently to remove the adhesive backing paper.
- 10 Prepare the inserter:** Place fingers on lined indentations. Pull up the center of the inserter until you hear a "click".
- 11** Carefully remove the needle guard by gently twisting and then pulling it off.
- Caution:** Before positioning the infusion set over the previously cleaned insertion site, place the tubing in the slot on the side of the inserter to ensure that the tubing is not caught underneath the device during insertion.
- 12 Insert Set:** Position device over the insertion site. Simultaneously press the round

indentations on both sides of the insertion device to insert the set.

- 13** Push gently on the center of the inserter to secure the adhesive onto the skin.
- 14** Remove the inserter and introducer needle by grasping the center of the inserter and pulling it gently back. Massage the tape thoroughly onto the skin. Fill the empty space in the cannula with the following amount of insulin:

U-100 insulin:

6 mm: 0.3 units (0.003 ml)

9 mm: 0.5 units (0.005 ml)

- 15** Push the lid back in place until you hear a "click."

To Disconnect

- A** This set allows you to temporarily disconnect from your pump without having to change the infusion set. Hold the adhesive in place by placing a finger just in front of the cannula housing, and gently squeeze the sides of the connector. Pull the connector needle straight out from the cannula housing.
- B** Insert the disconnect cover into the cannula housing until you hear a "click".

To Reconnect

- C** Prime the infusion set until medication dispenses from the needle.
- Caution:** When priming the infusion set, hold the needle pointing down.
- D** Place a finger on the adhesive tape just in front of the cannula housing.
- E** Push the connector needle straight in until you hear a "click".

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Medtronic Ltd.
Medtronic Österreich GmbH
Medtronic B.V. • 6422 PJ Heerlen • The Netherlands