

Medical emergency card

I am wearing a required medical device prescribed by my physician.



Medtronic

In an emergency, please contact:

physician name:

phone number:

family member:

phone number:

I have diabetes.

If my behavior is peculiar (appear intoxicated) or I am unconscious, it may be a result of severe low blood sugar.

I am not intoxicated.

Call a medical team for assistance.

If I am able to swallow, give me a source of sugar, (examples: juice, candy, non-diet soft drink).

Healthcare Professionals

I am wearing an external insulin pump.

- The pump is delivering fast-acting insulin at a constant rate.
- To stop insulin delivery, press ACT, use down arrow button to select Suspend, press ACT. Follow instructions on the screen.
- Do not remove the pump battery.
- Do not remove the pump without medical consent.
- If the pump is alarming, follow the instructions on the pump screen or call the local Medtronic Diabetes representative listed below.

Local Medtronic Diabetes Representative

name:

phone number:

Airport information

I am wearing a required medical device prescribed by my physician.



Medtronic

This continuous glucose monitoring (CGM) feature on your pump is safe for use on U.S. commercial airlines. If questioned by airline personnel about use of your Medical-Portable Electronic Device (M-PED), show them this card. If the airline flight crew still request that you turn it off, you must comply.

The transmitter is an M-PED with emission levels that meet RTCA/DO160, Section 21, Category M. Per FAA Advisory Circular #91-21.1B, dated 8/25/06, any M-PED that meets this standard in all modes “may be used onboard the aircraft without any further testing by the operator”.



This device can withstand exposure to common electrostatic (ESD) and electromagnetic interference (EMI), including common security systems such as an airport metal detector. During exposure, it can remain connected and powered on.

Medtronic has conducted official testing on the effects of the **full body scanners at airports** with Medtronic medical devices. Some of the new scanners may include x-ray. **To avoid removing your devices, you may request an alternative screening process.** If you choose to go through a full body scanner, you must remove your insulin pump and CGM (sensor and transmitter). Do not send your devices through the x-ray machine as an alternative.

Patient information

my name:

my address:

home phone number:

Medical Device Information

medical device type:

device serial number:



Medtronic

Medtronic MiniMed
Northridge, CA 91325
USA
800 646 4633
818 576 5555