# WALLET CARD

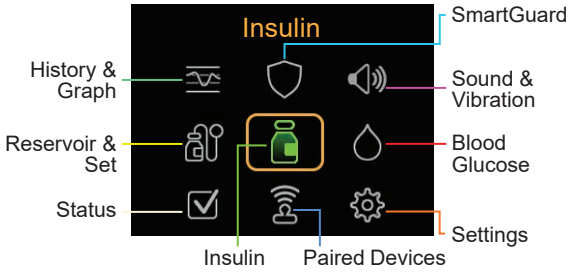
MiniMed™ 780G System

# Medtronic

**Medtronic MiniMed**  
18000 Devonshire Street  
Northridge, CA 91325 USA  
1800 646 4633  
+1818 576 5555  
[www.medtronicdiabetes.com](http://www.medtronicdiabetes.com)


M024356C003\_1

## Main Menu

<b>History &amp; Graph</b> History Sensor Glucose Review* Graph Time in Range*	<b>SmartGuard</b> SmartGuard Checklist Temp Target* SmartGuard Settings* SmartGuard*
<b>Sound &amp; Vibration</b> Silence Sensor Alerts* Volume Sound Vibration Alert Settings	<b>Reservoir &amp; Set</b> New Reservoir & Set Fill Cannula
<b>Insulin</b> Bolus Basal Suspend All Delivery Delivery Settings	<b>Blood Glucose</b> Enter BG
<b>Status</b> Suspend All Delivery SmartGuard Checklist Pump Sensor*	<b>Paired Devices</b> Pair New Device Pair CareLink Mobile Meter Sensor
<b>Settings</b> Alert Settings Delivery Settings Device Settings	<i>*In some cases, menu choices appear, but are greyed out if certain settings are turned on or off.</i>

## Home screen in SmartGuard™ feature

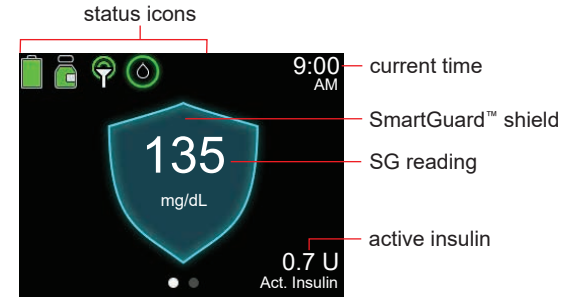
## Stopping and resuming insulin delivery

<b>Stopping all insulin delivery</b> The Suspend All Delivery option stops all insulin delivery (basal and bolus).	<b>To stop all insulin delivery:</b> 1. From the Home screen, press  and select . 2. Select <b>Suspend All Delivery</b> . 3. Select <b>Yes</b> when prompted.
<b>Resuming basal insulin delivery</b> The Resume Basal option restarts the basal delivery after the pump has been suspended. The Resume Basal option does not restart a bolus delivery.	<b>To resume basal insulin delivery:</b> 1. From the Home screen, press  and select . 2. Select <b>Resume Basal</b> . 3. Select <b>Yes</b> when prompted. <b>NOTE:</b> Basal insulin delivery can be manually resumed during a Suspend before low or Suspend on low event.
<b>Stopping a bolus delivery</b> The Stop Bolus option stops a bolus delivery only. The Stop Bolus option does not stop the basal delivery.	<b>To stop a normal bolus, or the Now portion of a Dual Wave™ bolus:</b> 1. From the Home screen, press  and select . 2. Select <b>Stop Bolus</b> . 3. Select <b>Yes</b> when prompted. 4. Select <b>Done</b> . <b>To stop a Square Wave™ bolus or the Square portion of a Dual Wave™ bolus:</b> 1. From the Home screen, press  and select . 2. Select <b>Bolus</b> . 3. Select <b>Stop Bolus</b> . 4. Select <b>Yes</b> when prompted. 5. Select <b>Done</b> .
<b>Manually resuming basal insulin delivery during a Suspend before low or Suspend on Low event.</b> If unable to wait for the pump to automatically resume the basal insulin during a Suspend before low or Suspend on low event, manually resume the basal delivery.	<b>To manually resume basal insulin delivery:</b> 1. From the Home screen, press  and select . 2. Select <b>Resume Basal</b> . 3. Select <b>Resume Basal</b> again. 4. Select <b>Yes</b> when prompted.


© 2021 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

MiniMed™, SmartGuard™, Dual Wave™, and Square Wave™ are trademarks of Medtronic MiniMed, Inc.
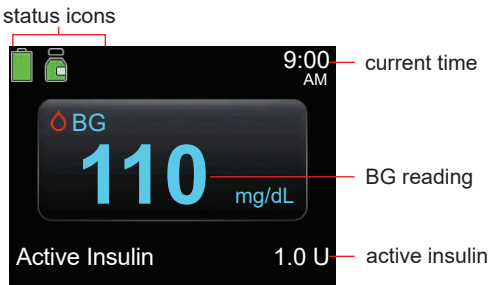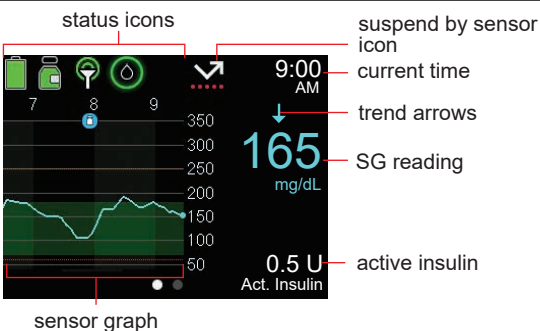
## Home screen without CGM in Manual mode

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