


Medtronic

How care partners can follow a pump patient's data using the MiniMed™ Mobile and CareLink™ Connect apps

Patients using the MiniMed™ Mobile app* and a compatible** pump can see their insulin pump and CGM data as well as receive alerts and notifications on their compatible* mobile device. This also allows them to share their data with up to 5 care partners via the CareLink™ Connect app, which enables care partners to view their data and receive alerts as well as notifications.

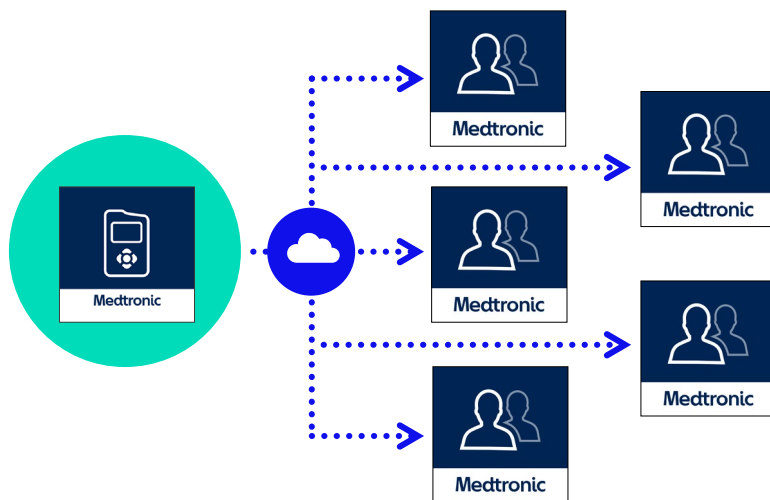


...➔ Indicates direction of data travel

1 MiniMed™ Mobile app user can have their data followed by up to 5 CareLink™ Connect app users.

Also, if you are a care partner for more than one person, the CareLink™ Connect app will allow you to follow up to 5 MiniMed™ Mobile app users.


1 CareLink™ Connect app* user can follow up to 5 MiniMed™ Mobile app users.

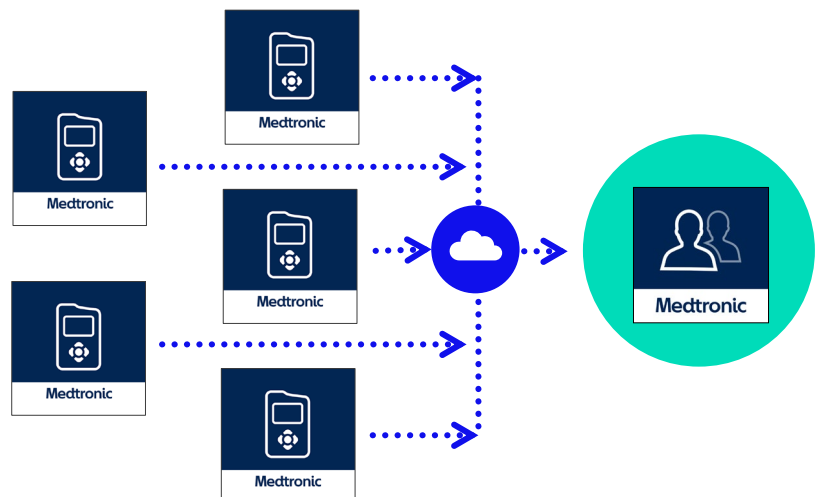

*Version 3.1 or higher

How to set up following

1 Make sure the pump user is using the MiniMed™ Mobile app with Sync to CareLink™ switched on and has their CareLink™ Personal username.

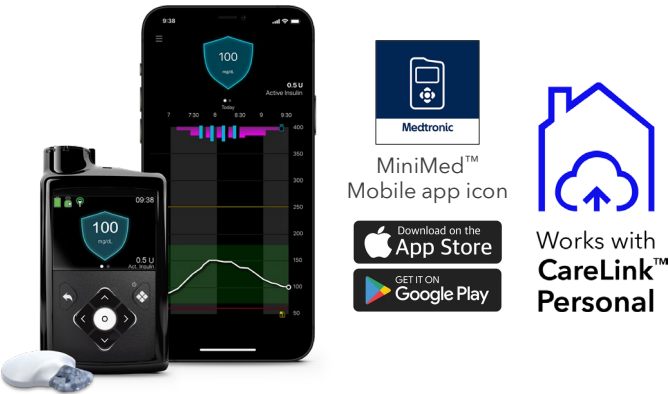

2 The care partner downloads and installs the CareLink™ Connect app (if not already done). They will need a CareLink™ Personal account also but the app will prompt them to sign up.



3 The care partner then enters the CareLink™ Personal username of the person they wish to follow. The MiniMed™ Mobile user has 24 hours to respond to the request either in their app or in CareLink™ Personal. Once agreed to, data can be seen and notifications set.



Up to 5 usernames can be added to follow up to 5 accounts

*Sync to CareLink™ feature must be ON
***Check your device compatibility at:
<https://www.medtronicdiabetes.com/app-check>



US-IPT-2300416 ©2023 Medtronic. Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic. Third party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company. The CareLink™ Connect app is intended to provide a secondary display of insulin pump and CGM (Continuous Glucose Monitor) data on a supported mobile device. CareLink™ Connect app is not intended to replace the real-time display of insulin pump or CGM data on the primary display device. All therapy decisions should be based on the primary display device. CareLink™ Connect app is not intended to analyze or modify the insulin pump and CGM data that it receives. Nor is it intended to control any function of the insulin pump or CGM system to which it is connected. CareLink™ Connect app is not intended to receive information directly from the insulin pump or CGM system.