

MAIN MENU



MAIN MENU



MANUAL MODE

- BOLUS
- ENTER BG
- BASAL
- AUDIO OPTIONS
- STATUS
- SUSPEND DELIVERY
- OPTIONS

AUTO MODE

- BOLUS
- ENTER BG
- TEMP TARGET
- AUDIO OPTIONS
- STATUS
- SUSPEND DELIVERY
- OPTIONS

**BOLUS**

- Bolus Wizard
- Manual Bolus
- Preset Bolus
- Delivery Settings

**BASAL**

- Temp Basal
- Preset Temp
- Basal Patterns
- Delivery Settings

**OPTIONS**

<p><b>Smart Guard</b></p> <ul style="list-style-type: none"> <li>Auto Mode                             <ul style="list-style-type: none"> <li>Auto Mode (On/Off)</li> <li>Auto Mode BG Alert (On/Off)</li> </ul> </li> <li>High Setup                             <ul style="list-style-type: none"> <li>Alert before High (On/Off)</li> <li>Time before High</li> <li>Alert on High (On/Off)</li> <li>Rise Alert (On/Off)</li> <li>Rise Limit</li> </ul> </li> <li>Low Setup                             <ul style="list-style-type: none"> <li>Alert before Low (On/Off)</li> <li>Alert on Low (On/Off)</li> <li>Suspend before Low (On/Off)</li> <li>Suspend on Low (On/Off)</li> <li>Resume basal alert (On/Off)</li> </ul> </li> <li>Snooze                             <ul style="list-style-type: none"> <li>High Snooze</li> <li>Low Snooze</li> </ul> </li> </ul> <p><b>History</b></p> <ul style="list-style-type: none"> <li>Summary</li> <li>Daily History</li> <li>Alarm History</li> <li>Sensor Glucose Review</li> <li>ISIG History</li> </ul> <p><b>Reservoir &amp; Tubing</b></p> <ul style="list-style-type: none"> <li>New Reservoir</li> <li>Fill Cannula</li> </ul>	<p><b>Delivery Settings</b></p> <ul style="list-style-type: none"> <li>Bolus Estimate Setup                             <ul style="list-style-type: none"> <li>Carb Ratio</li> <li>Bolus Wizard (On/Off)</li> <li>Insulin Sensitivity Factor</li> <li>BG Target</li> <li>Active Insulin Time</li> </ul> </li> <li>Basal Pattern Setup</li> <li>Preset Temp Setup</li> <li>Preset Bolus Setup</li> <li>Dual/Square Wave</li> <li>Bolus Increment</li> <li>Max Basal/Bolus</li> <li>Easy Bolus</li> <li>Auto Suspend</li> <li>Bolus Speed</li> </ul> <p><b>Event Markers</b></p> <ul style="list-style-type: none"> <li>BG</li> <li>Injection</li> <li>Food</li> <li>Exercise</li> <li>Other</li> </ul> <p><b>Reminders</b></p> <ul style="list-style-type: none"> <li>Personal</li> <li>Bolus BG Check</li> <li>Missed Food Bolus</li> <li>Low Reservoir</li> <li>Set Change</li> <li>Calibration</li> </ul>	<p><b>Utilities</b></p> <ul style="list-style-type: none"> <li>Sensor Settings                             <ul style="list-style-type: none"> <li>Sensor (On/Off)</li> <li>Sensor Connections</li> <li>Calibrate sensor</li> </ul> </li> <li>Airplane Mode</li> <li>Display Options                             <ul style="list-style-type: none"> <li>Brightness</li> <li>Backlight</li> </ul> </li> <li>Time &amp; Date</li> <li>Remote Bolus</li> <li>Block</li> <li>Self Test</li> <li>Manage Settings                             <ul style="list-style-type: none"> <li>Save Settings</li> <li>Restore Settings</li> <li>Clear All Settings</li> <li>Clear Active Insulin</li> <li>Settings History</li> </ul> </li> <li>Sensor Demo</li> <li>Device Options                             <ul style="list-style-type: none"> <li>Manage Devices</li> <li>Connect Device</li> </ul> </li> <li>Language</li> </ul>
--	---	---

**BOLUS**

- BG
- Carbs

**ENTER BG**

- BG Entry

**TEMP TARGET**

- Duration
- Cancel Temp Target

**AUDIO OPTIONS**

- Alert Silence Options**
  - High Alerts Only
  - High & Low Alerts
  - All Sensor Alerts
  - Duration
- Audio (On/Off)**
- Vibrate (On/Off)**
- Volume (-/+)**

**STATUS**

- Auto Mode Readiness
- Notifications
- Quick Status
- Pump
- Sensor
- Settings Review

**SUSPEND DELIVERY**