



# WALLET CARD




MiniMed® 670G

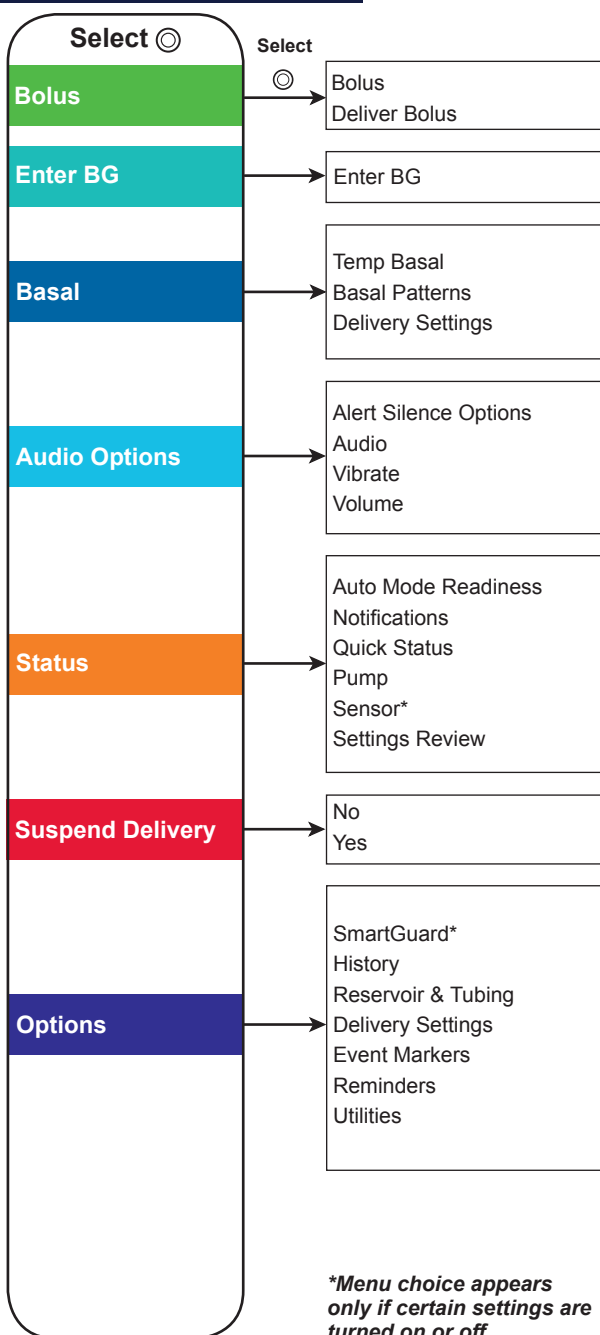


## Home screen without CGM in Manual Mode:

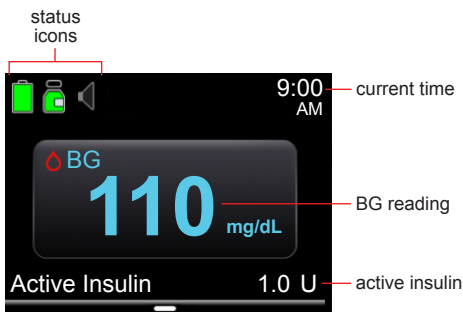

## Home screen with CGM in Manual Mode:

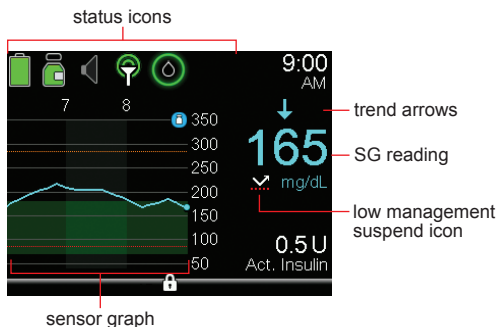

## Home screen in Auto Mode:

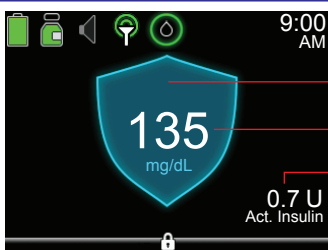

Auto Mode shield

SG value

active insulin

## Stopping and resuming insulin delivery

### Stopping all insulin delivery

The Suspend Delivery option stops all insulin delivery (basal and bolus).

### To stop all insulin delivery:

1. Go to the Suspend Delivery screen. Press **⊙** from the Home screen and select **Suspend Delivery**.
2. Select **Yes** when prompted.

### Resuming basal insulin delivery

The Resume option restarts your basal delivery after your pump has been suspended. The Resume option does not restart a bolus delivery.

### To resume your basal insulin delivery:

1. Press **⊙** from the Home screen and select **Resume Delivery**.
2. Select **Yes** when prompted.

**NOTE:** If a temp basal was active when you suspended your pump, it will resume if there is still time remaining.

### Stopping a bolus delivery

The Stop Bolus option stops a bolus delivery only. The Stop Bolus option does not stop your basal delivery.

### To stop an Auto Mode bolus, a Manual Mode Normal bolus, or the Now portion of a Dual Wave<sup>®</sup> bolus:

1. Press **⊙** from the Home screen and select **Stop Bolus**.
2. Select **Yes** when prompted.
3. Select **Done**.

### To stop a Square Wave<sup>®</sup> bolus or the Square portion of a Dual Wave bolus:

1. Press **⊙** from the Home screen and select **Bolus**.
2. Select **Stop Bolus**.
3. Select **Yes** when prompted.

### Manually resuming basal delivery during a suspend by sensor event

If you do not want to wait for your pump to automatically resume your basal insulin during a Suspend before low or Suspend on low event, you can manually resume your basal delivery.

### To manually resume basal delivery:

1. Press **⊙** from the Home screen and select **Suspended before low** or **Suspended on low**. The SmartGuard<sup>™</sup> Suspend Status screen appears.
2. Select **Resume Basal**.
3. Select **Yes** when prompted.

©2016 Medtronic MiniMed, Inc. All rights reserved.

MiniMed<sup>®</sup>, Dual Wave<sup>®</sup>, and Square Wave<sup>®</sup> are registered trademarks of Medtronic MiniMed, Inc.

SmartGuard<sup>™</sup> is a trademark of Medtronic MiniMed, Inc.

# Medtronic



**Medtronic MiniMed**  
18000 Devonshire Street  
Northridge, CA 91325  
USA  
800 646 4633  
818 576 5555  
[www.medtronicdiabetes.com](http://www.medtronicdiabetes.com)

6026138-012\_a