WALLET CARD

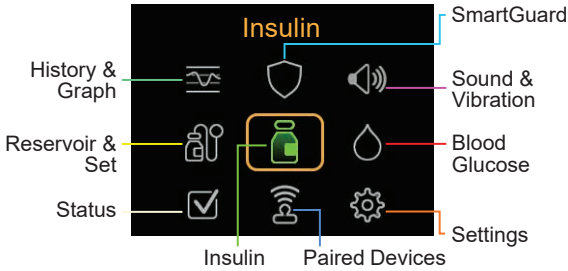
MiniMed™ 780G System

Medtronic

Medtronic MiniMed
18000 Devonshire Street
Northridge, CA 91325 USA
1800 646 4633
+1818 576 5555
www.medtronicdiabetes.com


M024356C002_1

Main Menu

<h3>History & Graph</h3> <p>History Sensor Glucose Review* Graph Time in Range*</p>	<h3>SmartGuard</h3> <p>SmartGuard Checklist Temp Target* SmartGuard Settings* SmartGuard*</p>
<h3>Sound & Vibration</h3> <p>Silence Sensor Alerts* Volume Sound Vibration Alert Settings</p>	<h3>Reservoir & Set</h3> <p>New Reservoir & Set Fill Cannula</p>
<h3>Insulin</h3> <p>Bolus Basal Suspend All Delivery Delivery Settings</p>	<h3>Blood Glucose</h3> <p>Enter BG</p>
<h3>Status</h3> <p>Suspend All Delivery SmartGuard Checklist Pump Sensor*</p>	<h3>Paired Devices</h3> <p>Pair New Device Pair CareLink Mobile Meter Sensor</p>
<h3>Settings</h3> <p>Alert Settings Delivery Settings Device Settings</p>	<p><i>*In some cases, menu choices appear, but are greyed out if certain settings are turned on or off.</i></p>

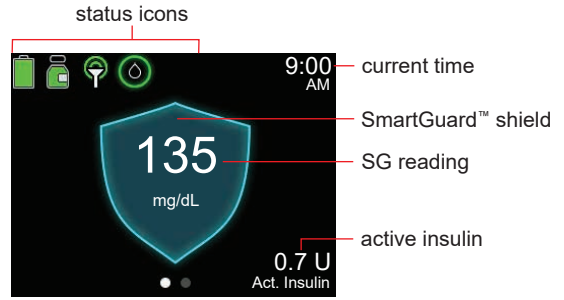Home screen in SmartGuard™ feature

Stopping and resuming insulin delivery

Stopping all insulin delivery

The Suspend All Delivery option stops all insulin delivery (basal and bolus).

To stop all insulin delivery:

1. From the Home screen, press and select .
2. Select **Suspend All Delivery**.
3. Select **Yes** when prompted.

Resuming basal insulin delivery

The Resume Basal option restarts the basal delivery after the pump has been suspended. The Resume Basal option does not restart a bolus delivery.

To resume basal insulin delivery:

1. From the Home screen, press and select .
 2. Select **Resume Basal**.
 3. Select **Yes** when prompted.
- NOTE:** Basal insulin delivery can be manually resumed during a Suspend before low or Suspend on low event.

Stopping a bolus delivery

The Stop Bolus option stops a bolus delivery only. The Stop Bolus option does not stop the basal delivery.

To stop a normal bolus, or the Now portion of a Dual Wave™ bolus:

1. From the Home screen, press and select .
2. Select **Stop Bolus**.
3. Select **Yes** when prompted.
4. Select **Done**.

To stop a Square Wave™ bolus or the Square portion of a Dual Wave™ bolus:

1. From the Home screen, press and select .
2. Select **Bolus**.
3. Select **Stop Bolus**.
4. Select **Yes** when prompted.
5. Select **Done**.

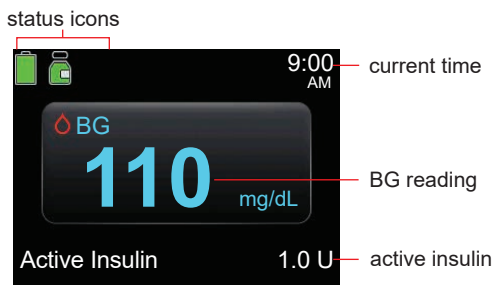
Manually resuming basal insulin delivery during a Suspend before low or Suspend on Low event.

If unable to wait for the pump to automatically resume the basal insulin during a Suspend before low or Suspend on low event, manually resume the basal delivery.


To manually resume basal insulin delivery:

1. From the Home screen, press and select .
2. Select **Resume Basal**.
3. Select **Resume Basal** again.
4. Select **Yes** when prompted.

Home screen without CGM in Manual mode

Home screen with CGM in Manual mode

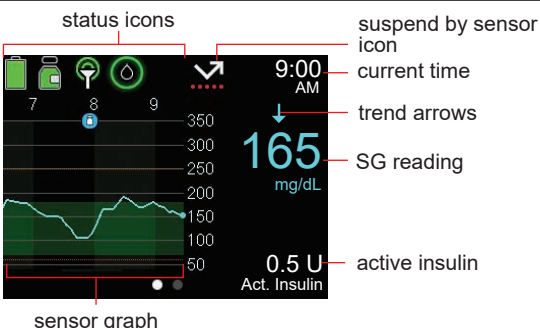© 2021 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

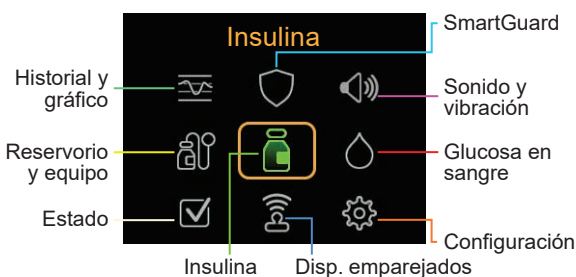
MiniMed™, SmartGuard™, Dual Wave™, and Square Wave™ are trademarks of Medtronic MiniMed, Inc.

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MiniMed™, SmartGuard™, Dual Wave™ y Square Wave™ son marcas comerciales de Medtronic MiniMed, Inc.