




1) Hold the Menu button until the meter turns on.

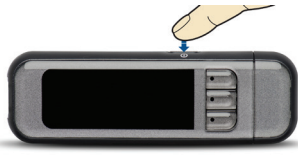

2) Scroll to your language and press **OK**. Press **OK** to confirm.

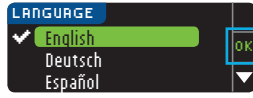

3) Press **OK** when asked **Connect to a MiniMed™ Pump?**

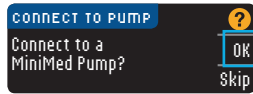


4) Press **OK**.



5) Press **Auto Connect**.

**Put the meter down and pick up your pump.**

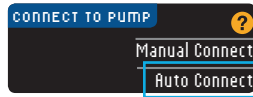

6) Press **O**.

7) Select **Options**.



8) Select **Utilities**.


9) Select **Device Options**.

10) Select **Connect Device**.

11) Select **Auto Connect** on your pump.




12) Press **v**.

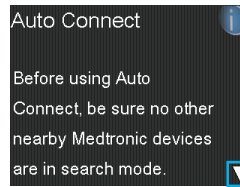


13) Press **>**.

14) Select **Continue**.


**Place the meter and pump next to each other.**

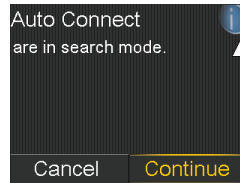

15) Select **Search** on your pump.

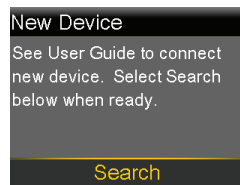

16) Select **Search** on your meter. *The search may take up to 2 minutes.*

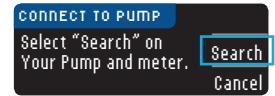


17) Check to see that the Device SN (serial number) on the pump screen matches the Device SN on the meter.




18) If they match select **Confirm** on the pump.


19) Check to see that the SN on the back of the pump matches the SN now on the meter screen.

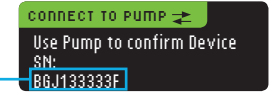

20) Select **Next** on the meter.

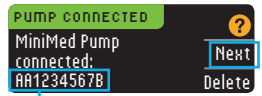

21) Press **Always**.\*

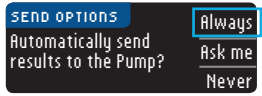

22) Then press **OK**.

23) Select the desired **Date Format**.




24) Press **OK**.

*If time and date are not correct you must change them on the pump.*




25) Press **Accept** to select **AutoLog is Off**.


*AutoLog allows you to mark a test result as Before Meal, After Meal or Fasting.\**

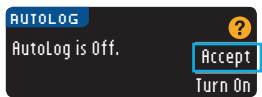


26) **Accept** or **Change** High and Low Alerts.\*

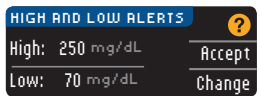

27) **Accept** or **Change** the Target Range.\* Press **Accept** again to confirm.

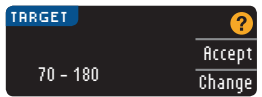

28) Setup is now complete and you are ready to use the meter.

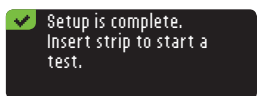

\*For more detail on this feature see your meter guide.