


Before you can begin using your new transmitter, you will need to delete your old transmitter from your pump. Make sure your new transmitter is fully charged and placed on the charger.

To delete your old transmitter:


- 1) Press **○**.
- 2) Select **Options**.
- 3) Select **Utilities**.
- 4) Select **Device Options**.
- 5) Select **Manage Devices**.

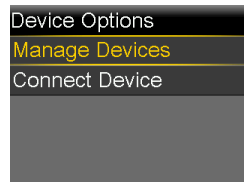


Locate the serial number that starts with "GT". This is your old transmitter.



- 6) Select the serial number for the old transmitter then select **Delete**.

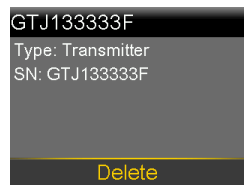

- 7) Select **Yes** to delete device.

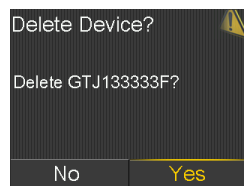


Now that your old transmitter serial number has been deleted from your pump, you can connect your new transmitter. Make sure your new transmitter is still attached to the charger.




To connect the new transmitter:


- 1) Press **○**.
- 2) Select **Options**.
- 3) Select **Utilities**.
- 4) Select **Device Options**.
- 5) Select **Connect Device**.
- 6) Select **Auto Connect**.

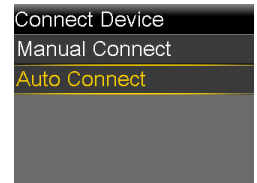


- 7) Press **∨**.

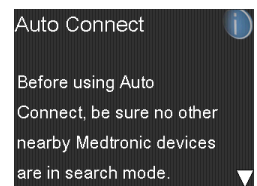

- 8) Select **Continue**.
- 9) Place the transmitter (still attached to the charger) next to the pump.

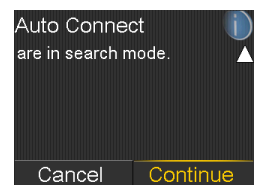


- 10) Select **Search** on your pump and immediately remove the transmitter from the charger.

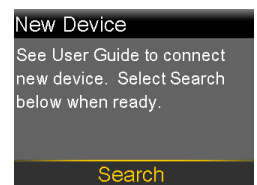

- 11) Once the device is found, confirm that the serial number (SN) shown on the pump is the serial number on the back of your transmitter.



- 12) If the serial number (SN) on the pump screen matches the back of the transmitter, select **Confirm**.

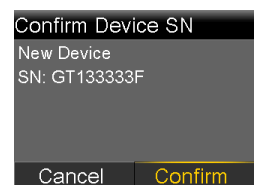

The new transmitter is now connected to your pump!