


1. Hold the Menu button until the meter turns on.



2. Scroll to **Setup** and click on **OK**.




3. Click **OK**.

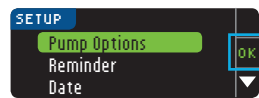


4. Click **Connection**.




5. Click **Delete**.

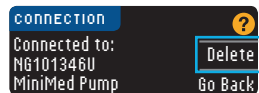


6. Click **Yes** to delete pump.

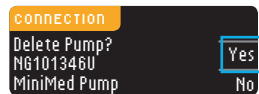


7. Wait...

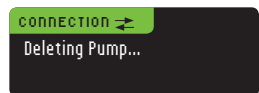


8. Click **OK**.

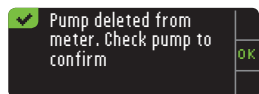


9. Click **OK**.

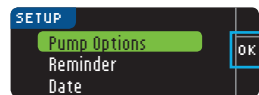


10. Click **Yes**.

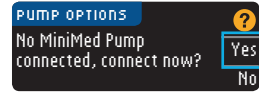

11. Click **OK**.

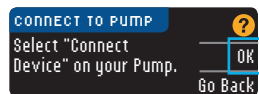


12. Press **Auto Connect**.

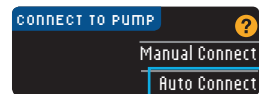

**Put the meter down and pick up your pump.**

13. Press **○**.

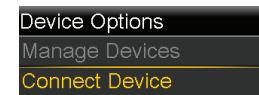

14. Go to **Options**.

15. Go to **Utilities**.


16. Go to **Device Options**.




17. Select **Connect Device**.

18. Select **Auto Connect** on your pump.

19. Press **∨**.

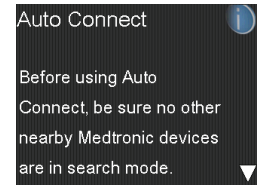


20. Select **Continue**.

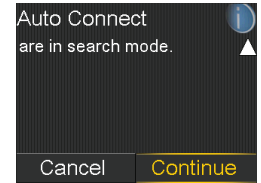


**Place the meter and pump next to each other.**


21. Select **Search** on your pump and meter. *The search may take up to 2 minutes.*

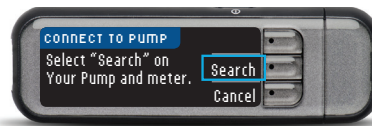


22. Check to see that the Device SN (serial number) on the pump screen matches the Device SN on the meter.

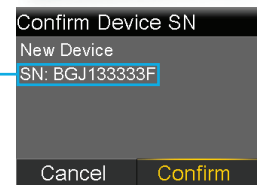

23. If they match, select **Confirm** on the pump.

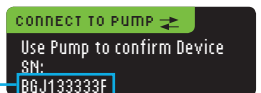

24. Check to see that the SN on the back of the pump matches the SN now on the meter screen.

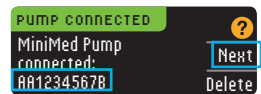

25. Follow the remaining steps on the meter to choose to **Always** send result to pump and select the **Date Format**.

**Setup is now complete.**

950M19718-011 05OCT2018 © 2018 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. Third party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company. PP-CNL-2.4-US-0553 Ascensia, the Ascensia Diabetes Care logo, and Contour are trademarks and/or registered trademarks of Ascensia Diabetes Care.