


#### **START HERE:**

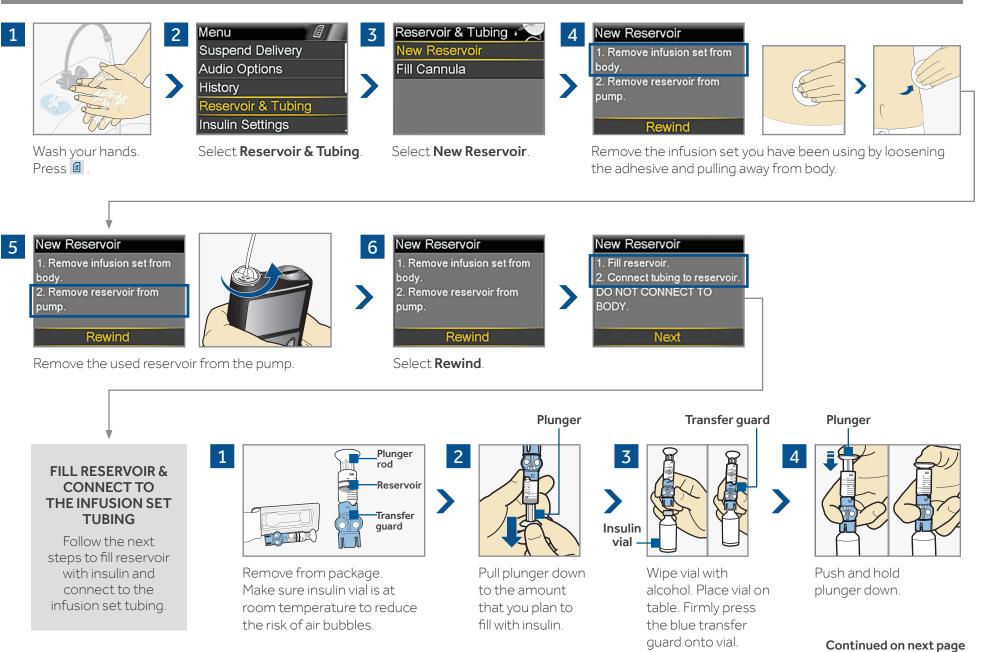


Warning: Do not use the reservoir or infusion set if any liquid gets on the top of the reservoir or inside the tubing connector. Liquid can temporarily block the vents and may result in the delivery of too little or too much insulin, which can cause hypoglycemia or hyperglycemia.

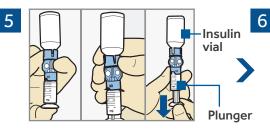


With your thumb still on the plunger, flip over so vial is on top. Release thumb and pull plunger down to fill with insulin

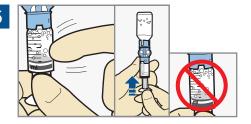

Tap the reservoir to move air bubbles to top of reservoir. Push plunger up to move air into vial.

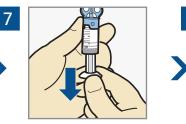

If needed, pull plunger back down to amount of insulin needed for 2-3 days.

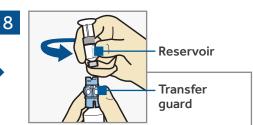

To avoid getting insulin on the top of the reservoir. turn vial over so it is upright. Hold transfer guard and turn reservoir counter-clockwise and remove from transfer quard.

2

#### Connector 2 3 1 CONNECT Insertion piece Site **RESERVOIR TO** connector **INFUSION SET** X You will place the Tubing tubing connector connector onto the end of the Gently push the tubing If you see air bubbles, tap infusion set to the reservoir to move them to connector onto reservoir. filled reservoir Turn clockwise until locked top. Push plunger just a bit Remove infusion set from package. You will hear a click. to move them into tubing. New Reservoir n ( 4 THE BACKLIGHT 1. Fill reservoir.

MAY HAVE 2. Connect tubing to reservoir. Plunger **TURNED OFF** DO NOT CONNECT TO BODY. Press any button to Active Insulin 0.0 ι turn the screen oad Reservoir Next back on. Twist plunger Continued on next page Select Load Reservoir and Select Next. counter-clockwise to unlock pump if necessary. loosen and remove

2

>


#### New Reservoir

3. Place reservoir into pump and lock. DO NOT CONNECT TO BODY.


Next



Place reservoir into pump.

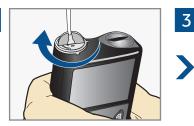

Turn clockwise until you feel reservoir lock into place.

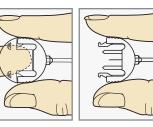

Press the edges of the site connector. With the other hand, pull the cap off.


New Reservoir

4

3. Place reservoir into pump and lock. DO NOT CONNECT TO BODY.


Next

Select Next.

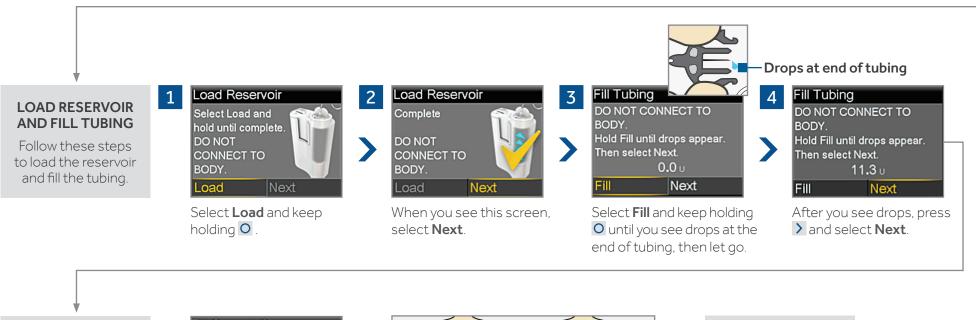


#### INSERT INFUSION SET (Two Methods)

Next, follow the steps to insert the infusion set into your body.

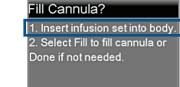

Fill Done

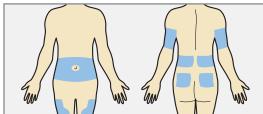


Choose an insertion site from the shaded areas shown here. Wipe with alcohol or other antiseptic.


### PROCEED TO METHOD 1 or METHOD 2

Continued on next page

### **METHOD 1:** By hand

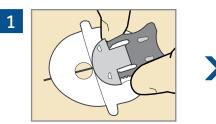


Hold the insertion piece by the gray needle handle using thumb and middle finger.

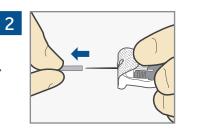

Hold the tape with your forefinger and remove the needle guard.




Pinch up skin at prepared site. Use other hand to insert needle at a 20-45 degree angle.

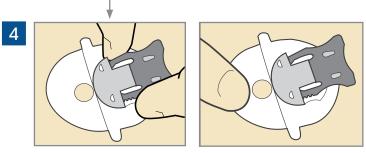

After it is inserted release the skin and hold inserted piece in place. Now go to the REMOVE INTRODUCER NEEDLE section on page 5.


2

**METHOD 2:** Using the serter

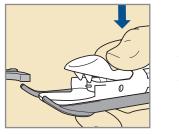

1

Use thumb to press down on the Lock & Release handle of the serter.

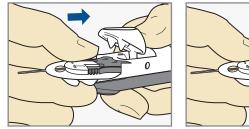




Place the insertion piece inside serter and let go of the Lock & Release handle. The serter prongs should fit securely into the holes of the insertion piece.


Continued on next page





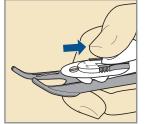


Use thumb to slide the Lock & Release handle back until you hear a click.

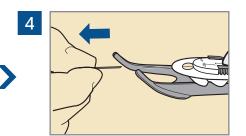


Gently pull off needle guard.



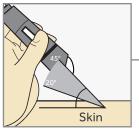

Rest arms of serter against skin so that it is at a 20-45 degree angle.



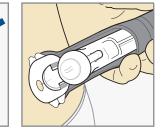

Press the white button at the top of serter to insert the infusion set.


1




Hold infusion set against skin with one hand. With the other hand press the Lock & Release handle to open serter. Then, slide serter back from infusion set.

#### REMOVE INTRODUCER NEEDLE

Next, follow the steps to remove the introducer needle.

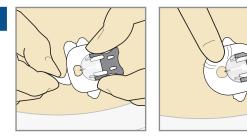


Peel paper from the front part of the adhesive and press adhesive securely to skin.

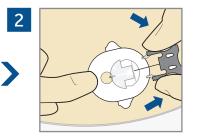


Squeeze arms on the gray needle handle and pull back to remove introducer needle.


DISPOSE NEEDLE INTO A SHARPS CONTAINER

>

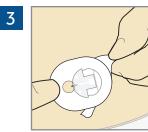

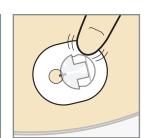


Peel rest of the paper without lifting infusion set too far from skin. Press adhesive securely to skin.

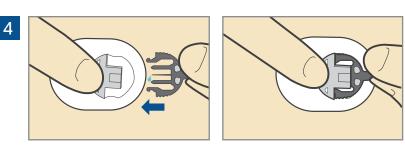

Connect infusion set tubing by sliding the site connector into the insertion piece on your skin.

### FILL CANNULA

You will now fill the cannula, the little tube under your skin, with insulin.

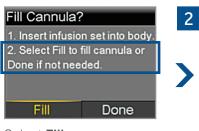

displays the insulin as it fills the cannula.

Select Fill.




Select **Fill amount** and enter 0.700. Then press **O**.



**Note:** Your pump will remember the **Fill amount** that you used last. Always verify that the Fill amount is correct.

- If it is correct, press ∨ to **Fill Now** and press **O**
- If it is incorrect, press O. Change to correct amount and press O.
  Select Fill Now.

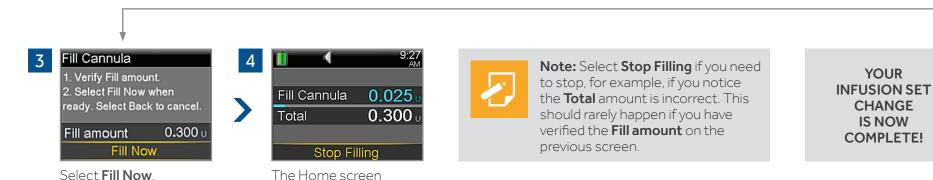

950M15863-011 20161001 ©2016 Medtronic MiniMed, Inc. All Rights Reserved.

6