

# Paradigm Revel™ Insulin Pump Settings Guide:

A guide for recording your current insulin pump settings for your reference.

## BASAL SETTINGS

### MAX BASAL RATE:

Main ⇒ Basal ⇒ Basal Setup ⇒ Max Basal Rate  
\_\_\_\_\_ u/h

### STANDARD BASAL RATES:

Main ⇒ Basal ⇒ Basal Setup ⇒ Set/Edit Basal

1: 12:00A \_\_\_\_\_ u/h  
2: \_\_\_\_\_ u/h  
3: \_\_\_\_\_ u/h  
4: \_\_\_\_\_ u/h  
5: \_\_\_\_\_ u/h  
6: \_\_\_\_\_ u/h

### TEMP BASAL TYPE:

Main ⇒ Basal ⇒ Basal Setup ⇒ Temp Basal Type  
 U/H  Percent of Basal

### BASAL PATTERNS:

Main ⇒ Basal ⇒ Basal Setup ⇒ Patterns  On  Off

### BASAL PATTERN A:

Main ⇒ Basal ⇒ Basal Setup ⇒ Patterns

1: 12:00A \_\_\_\_\_ u/h  
2: \_\_\_\_\_ u/h  
3: \_\_\_\_\_ u/h  
4: \_\_\_\_\_ u/h

### BASAL PATTERN B:

Main ⇒ Basal ⇒ Basal Setup ⇒ Patterns

1: 12:00A \_\_\_\_\_ u/h  
2: \_\_\_\_\_ u/h  
3: \_\_\_\_\_ u/h  
4: \_\_\_\_\_ u/h

## BOLUS SETTINGS

### MAX BOLUS:

Main ⇒ Bolus ⇒ Bolus Setup ⇒ Max Bolus  
\_\_\_\_\_ units

### EASY BOLUS:

Main ⇒ Bolus ⇒ Bolus Setup ⇒ Easy Bolus ⇒ On/Set  
 On/Set  Off \_\_\_\_\_ units

### DUAL/SQUARE OPTION:

Main ⇒ Bolus ⇒ Bolus Setup ⇒ Dual Square Bolus  
 On  Off

### BG REMINDER:

Main ⇒ Bolus ⇒ Bolus Setup ⇒ BG Reminder  
 On  Off

### BOLUS WIZARD SETUP:

Main ⇒ Bolus ⇒ Bolus Setup ⇒ Bolus Wizard Setup ⇒  
Edit Settings **WIZARD:**  On  Off

**ACTIVE INSULIN TIME:** \_\_\_\_\_ Hrs

**CARB UNITS:**  Grams  Exch

### CARB RATIOS:

1: 12:00AM \_\_\_\_\_ grams/U  
2: \_\_\_\_\_ grams/U  
3: \_\_\_\_\_ grams/U  
4: \_\_\_\_\_ grams/U  
5: \_\_\_\_\_ grams/U  
6: \_\_\_\_\_ grams/U

**BG UNITS:**  mg/dL  mmol/L

### INSULIN SENSITIVITY:

1: 12:00AM \_\_\_\_\_ mg/dL/U  
2: \_\_\_\_\_ mg/dL/U  
3: \_\_\_\_\_ mg/dL/U  
4: \_\_\_\_\_ mg/dL/U

### BG TARGETS/TARGET RANGES:

1: 12:00AM \_\_\_\_\_ - \_\_\_\_\_ mg/dL  
2: \_\_\_\_\_ - \_\_\_\_\_ mg/dL  
3: \_\_\_\_\_ - \_\_\_\_\_ mg/dL  
4: \_\_\_\_\_ - \_\_\_\_\_ mg/dL

## SENSOR SETTINGS

### SENSOR:

Main ⇒ Sensor ⇒ Edit Settings ⇒ Sensor  On  Off

### GLUCOSE ALERTS:

Main ⇒ Sensor ⇒ Edit Settings ⇒ Glucose Alerts  
 On  Off

**GLUCOSE LIMITS:** \_\_\_\_\_ - \_\_\_\_\_ mg/dL

### HI REPEAT:

Main ⇒ Sensor ⇒ Edit Settings Main ⇒ Glucose Alerts  
⇒ Glucose Limits ⇒ Hi Repeat ⇒ \_\_\_\_\_

### LO REPEAT:

Main ⇒ Sensor ⇒ Edit Settings Main ⇒ Glucose Alerts  
⇒ Glucose Limits ⇒ Lo Repeat ⇒ \_\_\_\_\_

**CAL REPEAT:** \_\_\_\_\_

**CAL REMINDER:** \_\_\_\_\_

**BG UNITS:**  mg/dL  mmol/L

**TRANSMITTER ID:** \_\_\_\_\_

**WEAK SIGNAL:** \_\_\_\_\_

## UTILITIES

### ALARM:

#### ALERT TYPE:

Main ⇒ Utilities ⇒ Alarm ⇒ Alert Type  Beep Long  
 Beep Medium  Beep Short  Vibrate

#### AUTO OFF:

Main ⇒ Utilities ⇒ Alarm ⇒ Auto Off  
 On  Off \_\_\_\_\_ Hrs

#### LOW RESERVOIR:

Main ⇒ Utilities ⇒ Alarm ⇒ low Resv Warning  
 \_\_\_\_\_ Units  \_\_\_\_\_ Time

**METER OPTIONS:**  On  Off

ID \_\_\_\_\_

Main ⇒ Utilities ⇒ Meter Options

**REMOTE OPTIONS:**  On  Off

ID \_\_\_\_\_

Main ⇒ Utilities ⇒ Remote Options