

Insulin Pump Settings Guide:

A guide for recording your current insulin pump settings so that they may be easily transferred to your new insulin pump.

BASAL SETTINGS

MAX BASAL RATE:

Main ⇒ Basal ⇒ Max Basal Rate
_____ u/h

STANDARD BASAL RATES:

Main ⇒ Basal ⇒ Basal Review

1: 12:00A _____ u/h
2: _____ u/h
3: _____ u/h
4: _____ u/h
5: _____ u/h
6: _____ u/h

TEMP BASAL TYPE:

Main ⇒ Basal ⇒ Temp Basal Type
 U/H Percent of Basal

BASAL PATTERNS:

Main ⇒ Basal ⇒ Patterns On Off

BASAL PATTERN A:

Main ⇒ Basal ⇒ Basal Review

1: 12:00A _____ u/h
2: _____ u/h
3: _____ u/h
4: _____ u/h

BASAL PATTERN B:

Main ⇒ Basal ⇒ Basal Review

1: 12:00A _____ u/h
2: _____ u/h
3: _____ u/h
4: _____ u/h

BOLUS SETTINGS

MAX BOLUS:

Main ⇒ Bolus ⇒ Max Bolus
_____ units

EASY BOLUS:

Main ⇒ Bolus ⇒ Easy Bolus
 On Off _____ units

DUAL/SQUARE OPTION:

Main ⇒ Bolus ⇒ Dual Square Bolus
 On Off

BG REMINDER:

Main ⇒ Bolus ⇒ BG Reminder
 On Off

BOLUS WIZARD SETTINGS

BOLUS WIZARD SETUP:

Main ⇒ Bolus ⇒ Bolus Wizard Setup ⇒
Review Settings

WIZARD: On Off**

CARB UNITS: Grams Exch

CARB RATIOS:

1: 12:00AM _____ grams/U
2: _____ grams/U
3: _____ grams/U
4: _____ grams/U
5: _____ grams/U
6: _____ grams/U

BG UNITS: mg/dL mmol/L

BOLUS WIZARD SETTINGS (CONT.)

INSULIN SENSITIVITY:

1: 12:00AM _____ mg/dL/U
2: _____ mg/dL/U
3: _____ mg/dL/U
4: _____ mg/dL/U

BG TARGETS/TARGET RANGES:

1: 12:00AM _____ - _____ mg/dL
2: _____ - _____ mg/dL
3: _____ - _____ mg/dL
4: _____ - _____ mg/dL

ACTIVE INSULIN TIME: _____ Hrs

**In Revel you must enter active insulin even when the Bolus Wizard is "Off."

SENSOR SETTINGS

SENSOR: On Off

Main ⇒ Sensor ⇒ Sensor Setup ⇒ Re-
view Settings

HIGH GLUCOSE: On Off

SET HIGH GLUCOSE: _____

HIGH SNOOZE: _____

LOW GLUCOSE: On Off

SET LOW GLUCOSE: _____

LOW SNOOZE: _____

ALARM SNOOZE: _____

CAL REMINDER: _____

BG UNITS: mg/dL mmol/L

TRANSMITTER ID: _____

MISSED DATA: _____

UTILITIES

ALARM:

ALERT TYPE:

Main ⇒ Utilities ⇒ Alarm ⇒ Alert Type

Beep Long Beep Medium

Beep Short Vibrate

AUTO OFF:

Main ⇒ Utilities ⇒ Alarm ⇒ Auto Off

On Off _____ Hrs

LOW RESERVOIR:

Main ⇒ Utilities ⇒ Alarm ⇒ low

Resv Warning _____ Units

_____ Time

METER OPTIONS: On Off

ID _____

Main ⇒ Utilities ⇒ Meter Options

REMOTE OPTIONS: On Off

ID _____

Main ⇒ Utilities ⇒ Remote Options

SENSOR SETTINGS (CONT.)

CHANGES WITH PARADIGM REAL-TIME REVEL™ INSULIN PUMP:

⇒ HIGH SNOOZE IS NOW HI REPEAT

⇒ LOW SNOOZE IS NOW LO REPEAT

⇒ ALARM SNOOZE IS NOW CAL REPEAT

⇒ MISSED DATA IS NOW WEAK SIGNAL