
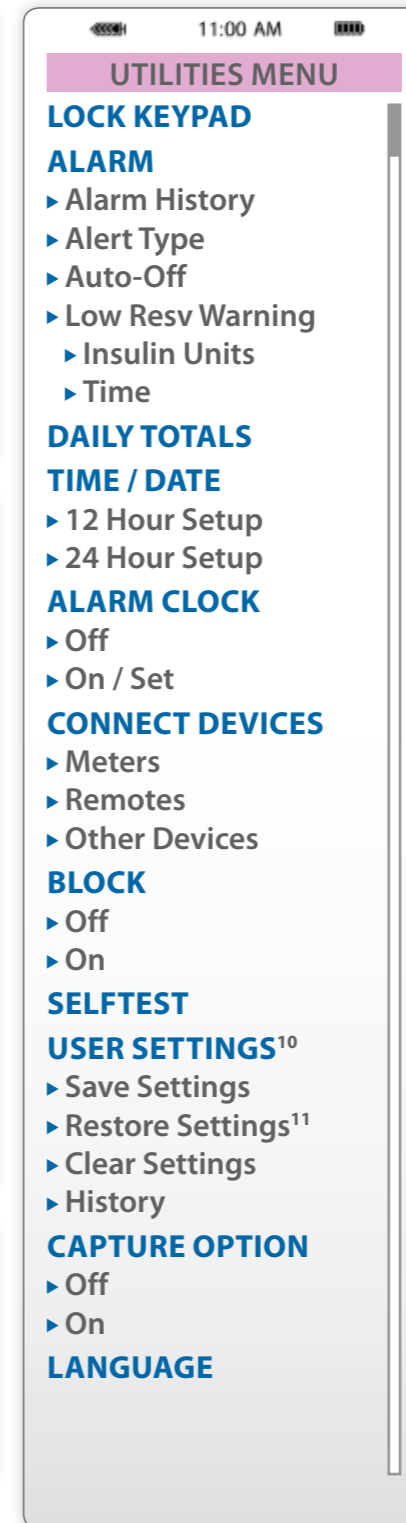
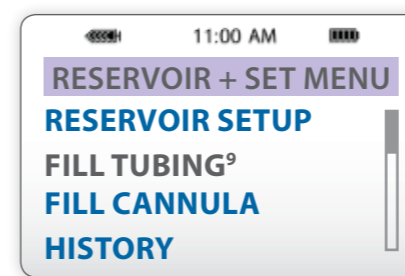
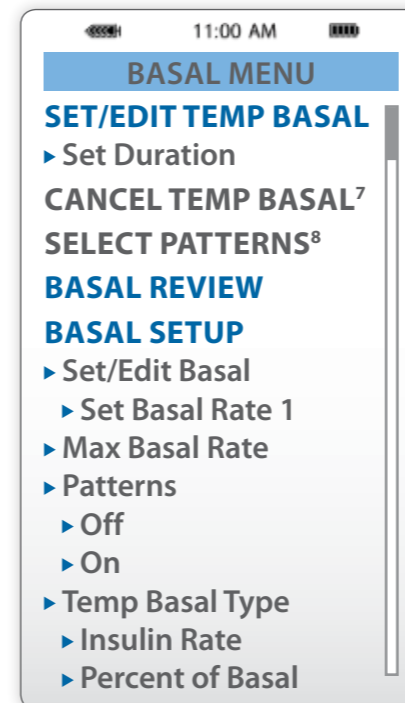
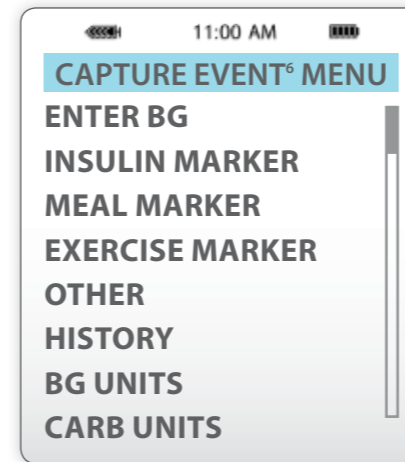
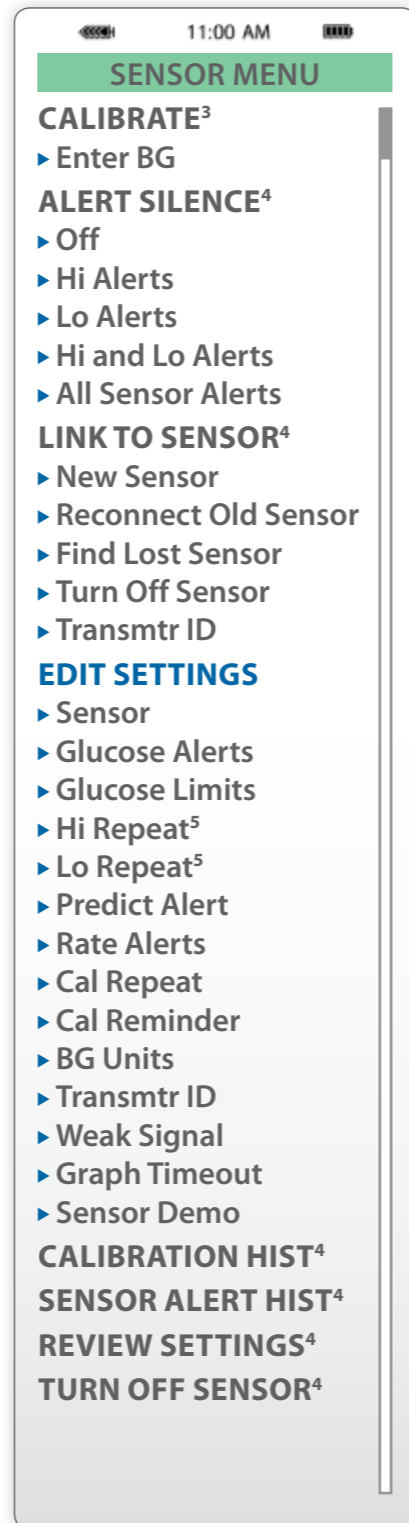
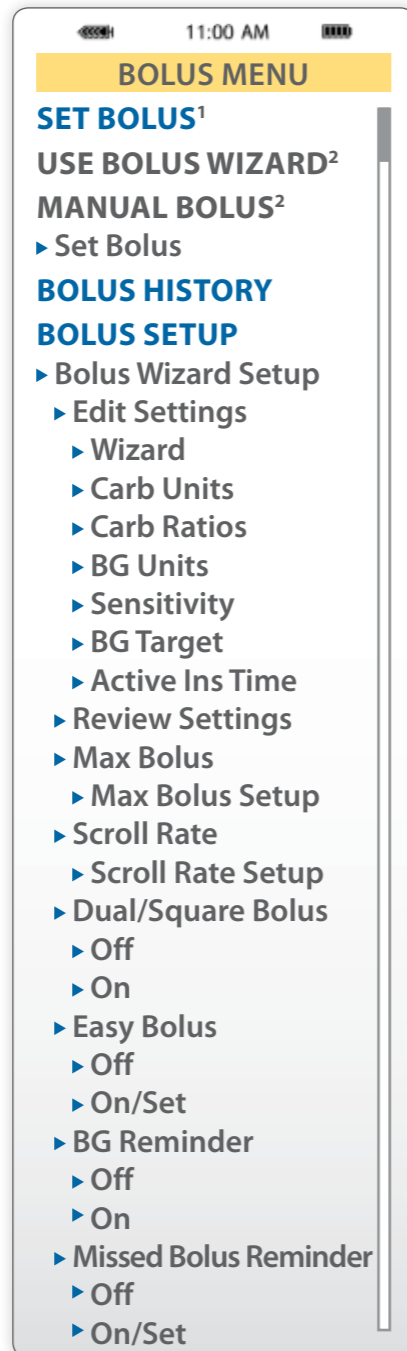



Appendix: Expanded Menu Map

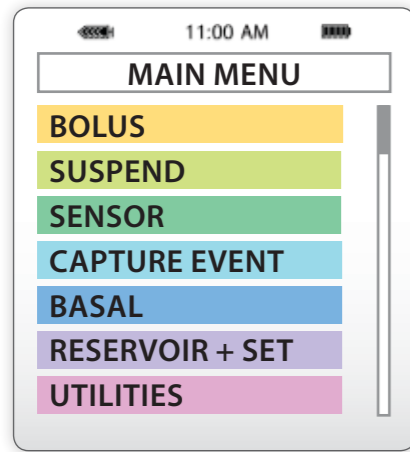


Status Screens

Press the button to go to the SENSOR STATUS screen.

Pump Status Screen

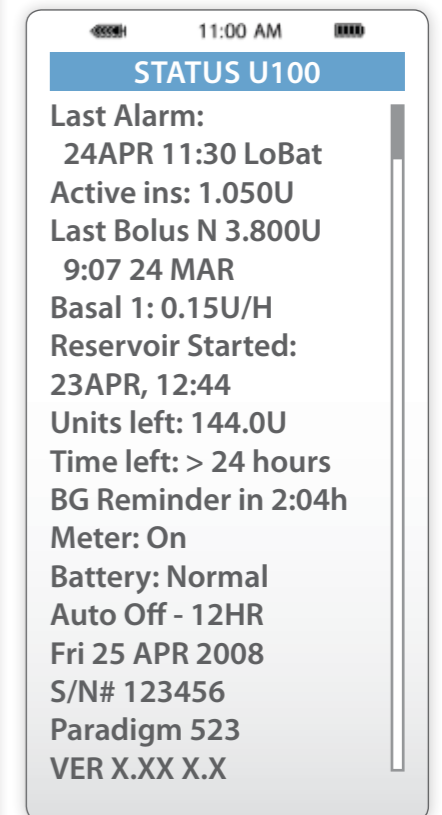

Sensor Status Screen

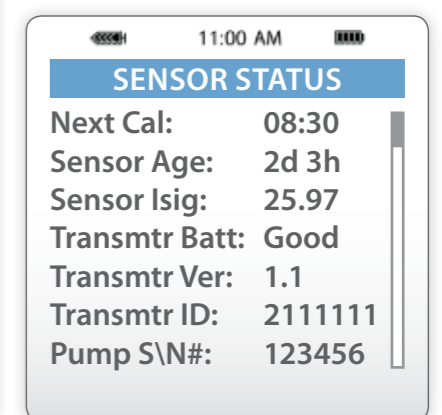

NOTE All screens are samples only. Actual screens depend on the current active functions. Blue Arrows (▶) indicate submenus.

¹ Displays only when the Bolus Wizard® feature is off.
² Displays only when the Bolus Wizard® feature is on.

³ Displays only when the sensor is communicating with the pump.
⁴ Displays if Sensor is on.
⁵ Displays if Glucose Alerts feature is on.

⁶ Displays if Capture Option is turned on in Utilities Menu.
⁷ Displays only after setting Temp Basal.
⁸ Displays only when the Patterns feature is on.
⁹ Displays only after a rewind.

¹⁰ Displays only when you hold and press .

¹¹ Displays only after the patient settings have been saved.