

What's new with the MiniMed™ 780G system†



Insulin pump software updates

Sensor life icon



Simplera Sync™ sensor: The sensor lasts up to 6 days, followed by a 24-hour grace period. When the sensor enters the grace period, **a red icon with a clock** will appear.



Guardian 4™ sensor: When one day remains in the life of the sensor, **a red icon with a 1** will appear.



When the sensor expires, **the icon turns solid black with an X**. Keep in mind, you can always see how many days of sensor life remain by viewing your Status screen.

Basal rate updates



Not for SmartGuard

A reminder that adjustments to this setting only impact insulin delivery when you are in Manual mode. This setting has no impact on SmartGuard™ technology.

Very high Basal setting

A message that appears when the system calculates that the programmed Manual mode basal pattern will deliver more insulin than you typically need.

SmartGuard™ Checklist



The **updated checklist** provides more details on how to enter the SmartGuard™ feature.

† The MiniMed™ 780G system is compatible with the Simplera Sync™ and Guardian™ 4 sensors.


Simplera Sync™ sensor insertion and pairing

All-in-one inserter, transmitter, and sensor.
No fingersticks with SmartGuard technology‡



Scan to view the sensor insertion video

Insertion site

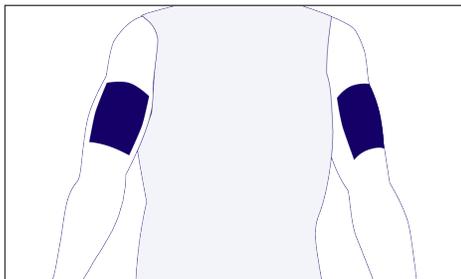

Ages 7 and older
Back of upper arm




The Simplera Sync™ sensor is indicated for **back of the upper arm** use only. Do not use the Simplera Sync™ sensor on the abdomen or other body sites including the buttocks, due to unknown or different performance that could result in hypoglycemia or hyperglycemia.

Sensor insertion



1. Select and clean insertion site.

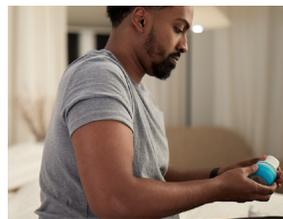

2. Inspect the inserter to ensure the tamper band is intact, then twist the cap to open.



3. Press the inserter firmly yet swiftly against the insertion site until you hear a click.




4. Use your finger to smooth down the adhesive.

Unpair current sensor

1. Go to **Main Menu** 2. Select **Paired Devices** 3. Select the **CGM device number** that corresponds to your Guardian™ transmitter 4. Select **Unpair** and confirm by selecting **Yes**

Pair the sensor

No unpairing is required with the Simplera Sync™ sensor. Pair your new one and the old one disconnects automatically.

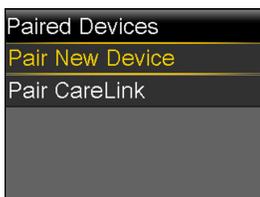


Go to **Main menu > Paired Devices > Pair New Device**



Select the **CGM device number that matches the serial number** located on the top of the inserter



Confirm the code on the screen matches the **CODE** on the top of the sensor inserter

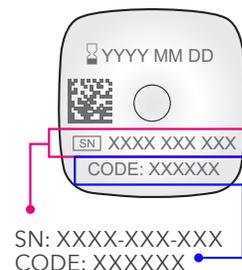

SN: XXXX-XXX-XXX
CODE: XXXXXX

Note: If the Simplera Sync™ sensor is not paired with the pump within 20 minutes after the cap is removed from the inserter, you will need to enter the CODE manually and select **Confirm** to pair the sensor.

‡ Fingersticks required in manual mode & to enter smartguard. If symptoms don't match alerts & readings, use a fingerstick. Refer to user guide. Pivotal trial participants spend avg of >93% in SmartGuard.

Important Safety Information for MiniMed 780G System:

MiniMed™ 780G System is for type 1 ages 7 and over. Prescription required. **WARNING: Do not use SmartGuard™ feature for people who require less than 8 units or more than 250 units of insulin/day.** For details, please visit www.medtronicdiabetes.com/support/download-library/user-guides and www.medtronicdiabetes.com/importantsafetyinformation for complete details.

US-DBA-2500381 © 2025 Medtronic. MiniMed and MiniMed logo are trademarks of Medtronic MiniMed, Inc. ™* Third-party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company.

Medtronic