

## Airport information



I wear a required medical device prescribed by my physician.

Medtronic



Medtronic

### Airport security

- Because travel rules are subject to change, it is advisable to check with the Transportation Safety Administration (TSA) before traveling. TSA information is found at <http://www.tsa.gov/travel/specialprocedures> or by calling 1-866-289-9673.
- The pump **must not** go through the x-ray machine that is used for carry-on or checked luggage.
- The full body scanner is also a form of x-ray. Remove the insulin pump and disconnect the infusion set at the insertion site before going through the full body scanner. If continuous glucose monitoring (CGM) is used, remove the sensor and transmitter before the scan.
- To avoid removing the devices, request an alternative screening process that does not use x-ray.
- The insulin pump, infusion set, reservoir, and CGM system can withstand exposure to airport metal detectors used at airport security checkpoints.

### In flight

The insulin pump, infusion set, reservoir, and CGM system are suitable for use in aircraft. When flying in an aircraft, it is important to keep the pump connected to the body and check glucose levels frequently.

### Patient information

Name:

Address:

Country:

Phone number (include country code):

### Medical device information

Medical device type:

Device serial number:

© 2021 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

Medtronic



Medtronic MiniMed  
18000 Devonshire Street  
Northridge, CA 91325 USA  
1800 646 4633  
+1 818 576 5555  
[www.medtronicdiabetes.com](http://www.medtronicdiabetes.com)

Rx Only

M013765C001\_2

# MEDICAL EMERGENCY CARD



Medtronic



I wear a required medical device prescribed by my physician.

Medtronic

[Blank header bar]

[Blank header bar]

[Blank header bar]
[Blank text area]
[Blank text area]
[Blank text area]

[Blank header bar]
[Blank text area]
[Blank text area]

## In an emergency, please contact

Physician name:
Phone number:
Emergency contact:
Phone number:

## I have diabetes

If my behavior is peculiar, if I appear intoxicated, or if I am unconscious, it may be a result of severe low blood sugar.

## I am not intoxicated Call for medical assistance

If I am awake and able to swallow, give me a source of sugar (for example: juice, candy, or non-diet soft drink). Do not try to give me food or drink if I am unconscious.

## Healthcare professionals

- I wear an insulin pump.
- The pump delivers rapid-acting insulin at a constant rate.
    - Press any button to wake the pump.
  - To stop insulin delivery on the pump:
    - From the home screen, press ^ to go to the status screen.
    - Press Ⓞ to **Suspend All Delivery**.
    - Press > to highlight **Yes**.
    - Press Ⓞ to suspend all insulin delivery.
  - **Do not** remove the pump battery.
  - **Do not** remove the pump without medical consent.
  - If the pump sounds an alarm, follow the instructions on the pump screen or contact a local Medtronic support representative for further assistance at 1-800-646-4633.